

Recaro Monza Reha 5-point Harness Instructions



1



2




3



4



5

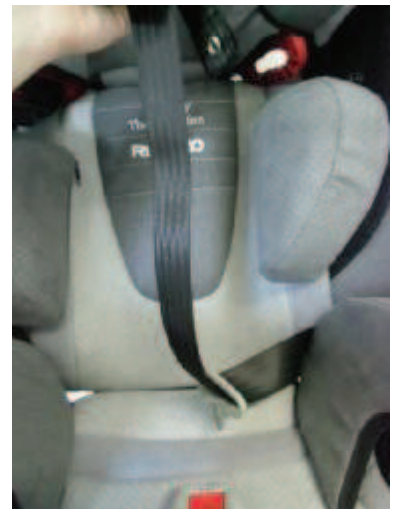

6



7



8



9

Removing the 5-point-harness

Loosen the Allen screws on the backside of the headrest (Fig. 1, arrow 1) and put the screws aside.

Loosen the Allen screws on the left and right hand side on the lower back (Fig. 1, arrow 2) and put the screws aside.

Loosen the Allen screw at the end of the belt which holds the buckle. (Fig. 4)
Now the 5-point-harness including the belt buckle can be removed from the Monza.

Mounting the 5-point-harness

Fix the belt with the Allen screw on both sides of the lower back. Ensure the plastic D-ring is pointing upwards on the right and left hand side of the backrest. (Fig. 3)

Lead the belt to the front between the backrest and seat on both sides.

Important: Never twist the belt!

Take the belt to the front and ensure the counterpart of the belt lock is hanging downside with its metal part. (Fig. 6)

Lead the ends of the shoulder belt (with two eyelets at the ends) through the belt guides on the headrest. (Fig. 7)

Fix the belt on the headrest with a Allen key. At the end of the belt there are two eyelets. You can vary the length of the belt by choosing the first or the second eyelet. Put the screw through the eyelet and fix it on the headrest. (Fig. 8)

Mount the belt lock with the red button in direction of travel. Take the Allen screw through the eyelet. The counterpart of the Allen screw is located in the seat of the Monza. Put the end of the belt through the thread in the seat upholstery, so that the eyelet lays over the cavity anchor. Fix the belt lock in direction of travel to the front with help of an Allen key in the cavity anchor. (Fig. 9)

Important: Ensure that all screws are fixed.