

EASyS Advantage, tRide, Swiftly & Swiftly 2



Wetterdach mit Beindecke

Montageanleitung – Zubehör



Sun and rain cover with leg blanket

Assembly instructions – Accessory



- EASyS Advantage
- tRide



EASyS Advantage (6637/6)



EASyS Advantage (6637/61)



- Swifty
- Swifty 2



EASyS Advantage (6638)



tRide (6637/6)



Swiftly (6637/6)



tRide (6637/61)



Swiftly (6637/61)



tRide (6638)



Swiftly (6638)

DE

Wetterdach mit Beindecke
Wetterdach
Klarsichtverdeck mit Beindecke, transparent

Art.-Nr.: 6637/6
Art.-Nr.: 6637/61
Art.-Nr.: 6638

Jeweils links und rechts Reißverschlüsse der Seitenpolster öffnen. (außer Swifty)
Die Dachaufnahmen (Kunststoffhalter) sind bereits rechts und links am Rahmen vormontiert.

Nun das Wetterschutzdach in die Halterung stecken. Reißverschlüsse so weit wie möglich zuziehen.

Die Beindecke kann zur Befestigung am Verdeck angeknöpft werden. Bei Bedarf kann die Verdeckschürze mittels umlaufendem Reißverschluss (unter hinterem Verdeckbügel) ganz entfernt werden.

ACHTUNG: Der Reha-Wagen / Reha-Buggy darf nicht mit montiertem Wetterdach zusammengefaltet werden.

GB

Sun and rain cover with leg blanket
Sun and rain cover
Sun and rain cover with leg blanket, transparent

Item code: 6637/6
Item code: 6637/61
Item code: 6638

Open the zippers each on the left and right lateral backrest upholstery. (except Swifty)

The receivers (plastic support brackets) are already mounted on the left and right of the frame.

Now put the sun / rain cover into the receiver. Close the zippers as far as possible.

The leg cover can be buttoned to the hood to secure it. If required, the hood apron can be completely removed using the hook and loop fastening (under the back hood frame).

CAUTION: The rehab stroller must not be folded up while the sun and rain cover is mounted.



Thomashilfen