

4.9: Accessories – Tray (Maximum Carrying Capacity 5kg)

The tray offers height, depth and angle adjustment, so that it can be adapted to suit your needs.

There are several styles of tray available (e.g. transparent, black, grey etc), the transparent style tray has been used for illustration purposes.

The tray comes supplied with the following:

- 2 fixed vertical stems (fitted to the tray)
- 2 depth adjustment brackets and thumbscrews (fitted to the tray)
- 2 Angle adjustable receivers and thumbscrews (fitted to the armrests of the Mini Tilt)



Fig 4.9.1

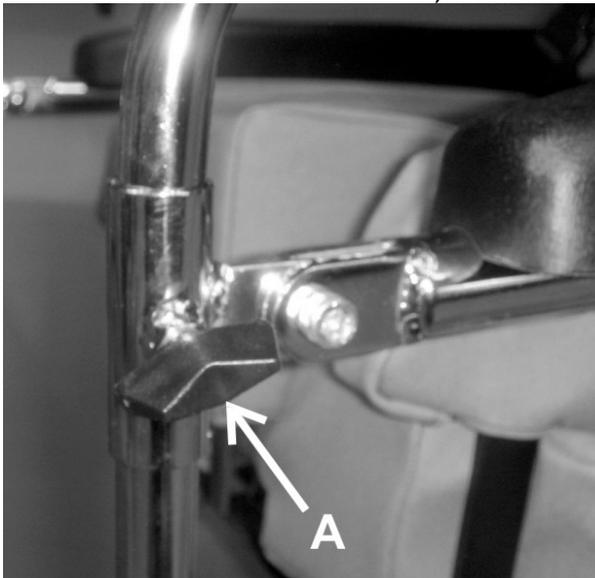


Fig 4.9.2

To fit the tray to the seat, locate the 2 vertical stems on the bottom of the tray into the 2 receivers on the armrests.

Position the tray to the required height, and then secure it in place by tightening the thumbscrews ('A' see left).

To adjust the angle of the tray, loosen the 2 angle adjuster screws 'B' (see right), using a 5mm hexagon key (not supplied).

Adjust the angle as required, and then lock in place by retightening the adjuster screws.

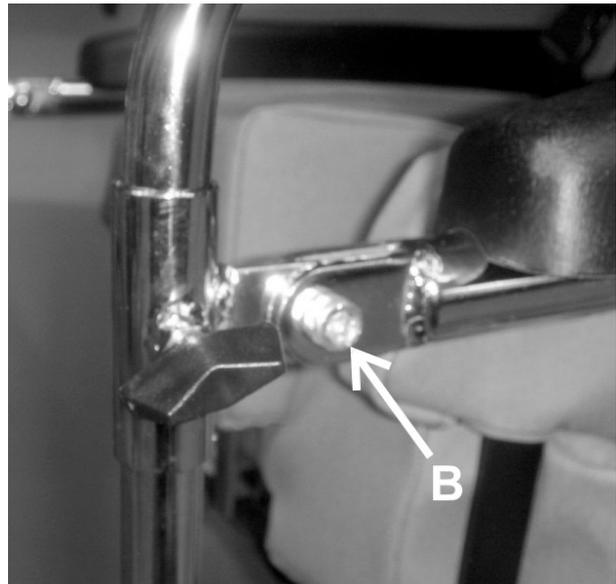


Fig 4.9.3

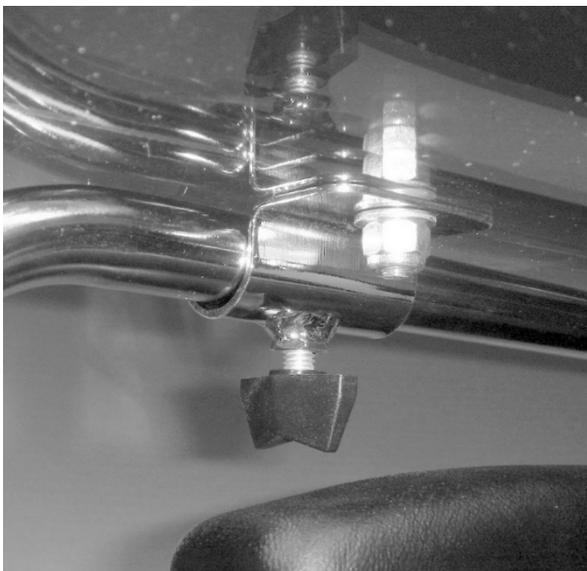


Fig 4.9.4

To adjust the depth of the tray, loosen the 2 thumbscrews under the base of the tray (see left).

Slide the tray in or out to the required depth, and then retighten the screws.

Notes:

- The tray must be removed when transporting the chair in specially adapted vehicles.
- The tray and armrests must be removed before folding the wheelchair.
- Never leave the occupant unattended with the tray fitted, especially when items are positioned on the tray.
- Extra care must be taken when placing food or drink on the tray.
- When operating the Tilt In Space facility, items must be removed from the tray to ensure they do not cause accidental harm to the occupant. The angle of the tray will need to be adjusted to allow for a new horizontal position (see above instructions for how to adjust the angle of the tray).