

6.7: Playtray – not suitable for rearward facing Snappi

The Snappi playtray offers height, depth and angle adjustment, so that it can be adapted to suit your needs. It is available in clear or black. (Clear playtray shown for illustrative purposes Fig 6.7.1).

The playtray comes supplied with the following:

- 2 angle adjustable vertical stems (marked A and B) in the bag.
- 2 depth adjustment brackets and thumbscrews (fitted to the tray, also marked A and B).
- 2 additional thumbscrews for the seat receiver brackets in the bag.



Fig 6.7.1

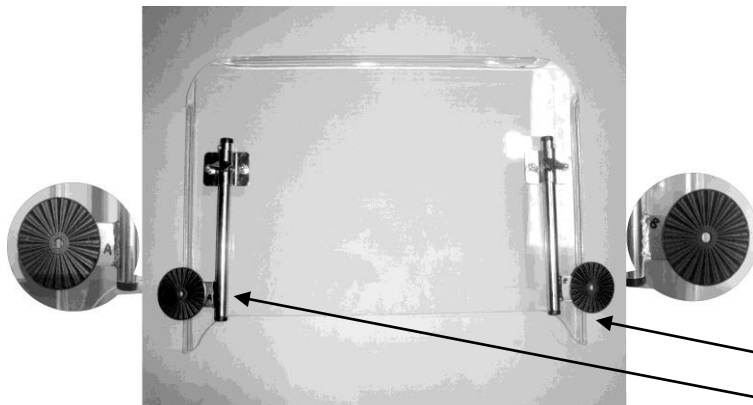


Fig 6.7.2

The playtray is supplied with the depth adjustment brackets fitted to the tray (as illustrated in Fig 6.7.2) These are marked with the letters A and B. A is one side, B is the other.

Fitted depth adjustment brackets and thumbscrews

The angle adjustable vertical stems will be in the bag and need to be fitted. When attaching the angle adjustable vertical stems to the depth adjustment brackets, most people should attach vertical stem A to depth adjustment bracket A (as illustrated in Fig 6.7.3) and then B to B.

However, if you require more outward adjustment (i.e. for the playtray to be further away from the occupant's torso), instead of attaching A to A and B to B, you can attach vertical stem A to depth adjustment bracket B and then vertical stem B to depth adjustment bracket A.

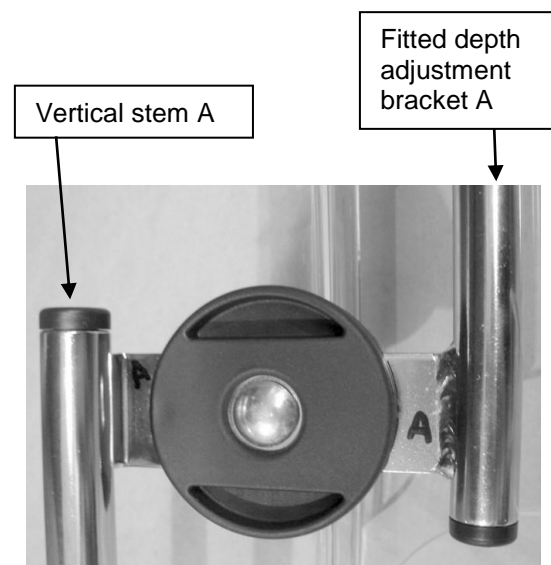


Fig 6.7.3



Fig 6.7.4

To fit the playtray to the seat, locate the 2 vertical stems on the bottom of the tray into the 2 receivers on the seat unit. Note there may be a small plastic cap in the top of the receiver. If this is the case, remove the cap using a small flat bladed screwdriver before fitting the playtray.

Screw the 2 thumbwheels supplied with the tray into the welded nut on the receivers as shown (see Fig 6.7.4). Position the tray to the required height, and then secure it in place by tightening the thumbscrews.

To adjust the angle of the playtray, loosen the 2 angle adjusters (see Fig 6.7.5).

Once loose, adjust the angle as required and retighten.

N.B. the angle adjusters are toothed, and may require some force to move. The angle adjusters move in 5-degree increments.



Fig 6.7.5



Fig 6.7.6

To adjust the depth of the playtray, loosen the 2 thumbscrews under the base of the tray (See Fig 6.7.6).

Slide the tray in or out to the required depth, and then retighten the screws.

Notes:

- We recommend that the playtray is removed before making any adjustments to the seat or seat tilt, and is then refitted and adjusted accordingly once the seat is in the desired position. When adjusting the Tilt in Space facility, the angle of the playtray must be adjusted to the new horizontal position (see above instructions for how to adjust the angle of the tray).
- All items must be removed from the playtray before making any adjustment to the tray or seat to avoid the risk of causing accidental harm to the occupant.
- The playtray must be removed before folding the pushchair.
- The playtray must be removed when transporting the chair in specially adapted vehicles.
- The playtray cannot be used at the same time as the rain cover.
- Never leave you child unattended with the play tray fitted, especially when items are positioned on the tray.
- Extra care must be taken when placing food or drink on the tray.
- Maximum carrying capacity of the play tray is 5kg.