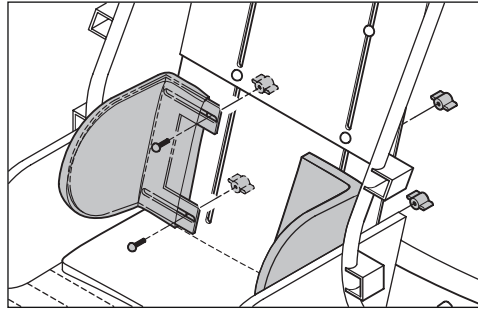
Swiftly EASyS Accessories Instructions

INDEX

Page	Accessory
2	Lateral Trunk Supports, Stiff
3	Umbrella
4/5	2-Point Pelvic Belt & 4-Point Pelvic Belt
6	Frame Padding
7/8	Grip Rail
9	Rain Shield For Sun And Rain Cover (Transparent)
10	Rain Cape
11/12	Sun and Rain Cover With Leg Blanket
13	Soft Pelvic Harness
14/15	H-Belt, Butterfly / Chest Harness and Seat Waistcoat
16	Basket
17	Head Pillow
18	Sleeping Bag
19	Tip-Assist
20	Lateral Trunk Supports: Flexible with chest belt
21	Foot Straps
22	Abduction Block

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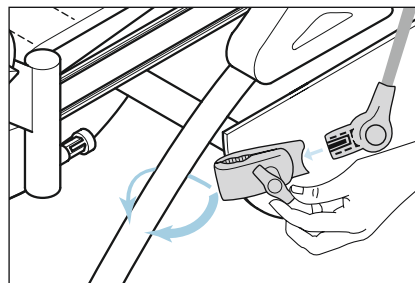
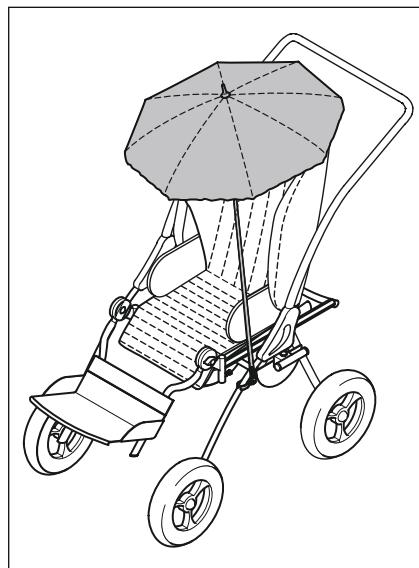
Open the back padding in the quilted area at the measured thorax height. Insert both button-headed screws through the grooves of the lateral trunk support, guide the lateral trunk support through the back and attach them to the backrest's longitudinal slots using the hand wing screws. Washers can also be used. The more sloping side of the lateral trunk support must be pointed upwards.

TIP: If there is any restricted operability for the wing screws, use the nuts enclosed with the packaging.

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GB Umbrella

Item code: 6594

The umbrella can be mounted either to the left or the right side of the chassis. Firstly detach the fixation clamp from the umbrella, unscrew it, and attach it to the frame at the desired height. Then put the umbrella into the fixation clamp (audibly clicks into place). The angle can be adjusted with the press button.

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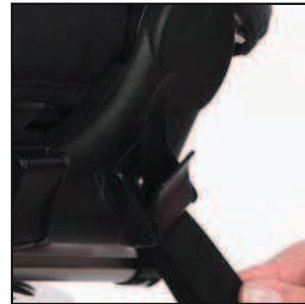
(GB)	2-point pelvis belt, with front pull	
	size 1, Item code: 8420	size 2, Item code: 8470
	4-point pelvis belt, with front pull	
	size 1, Item code: 8421	size 2, Item code: 8471

The two **bottom belt ends** can be attached with cam buckles to either the back plate/back bow or the plastic element at the side (fig. 1-3). Guide the belts left and right next to the back upholstery to the corresponding cam buckle and fix them.

The front belts of the 4-point pelvic belt are attached right and left at the plastic element with a cam buckle, as well. Guide the belt through the slot in the width plate, insert it into the cam buckle, and fix it (Fig. 4).

Swiftly (only 2-point pelvis belt)

The two **bottom belt ends** can be attached with cam buckles to the back plate (Fig. 5). Guide the belts left and right next to the back upholstery to the corresponding cam buckle and fix them.

Tip: The belt length can also be adjusted from the front. Pull the D-ring to shorten the belt.

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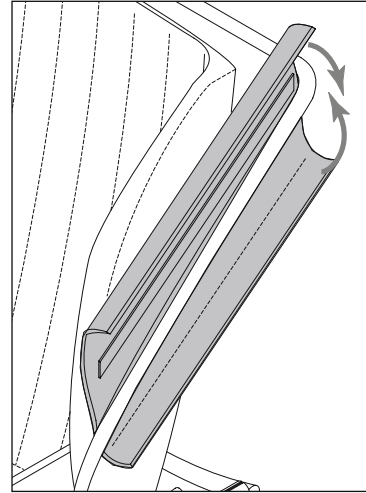
GB Frame padding

Item code: 6622

Lay the frame padding with the Velcro around the lateral push bars of the rehab stroller (wider side of the padding faces downwards) and close the Velcro.

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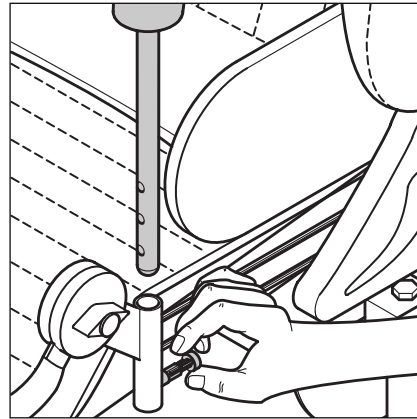
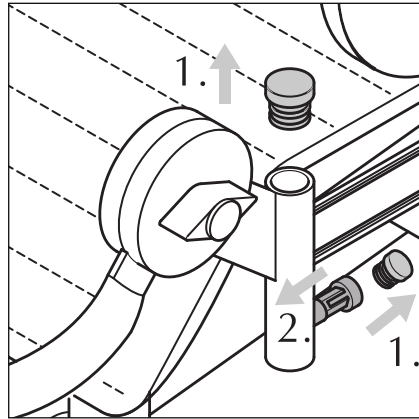
ⒸGB	ThevoTwist		
	Grip rail	size 1, Item code: 6372	size 2, Item code: 6572
	Grip rail, heavily bent	size 1, Item code: 6372/1	
	EASyS		
	Grip rail	size 1, Item code: 6621	size 2, Item code: 6721
	Grip rail, heavily bent	size 1, Item code: 6621/1	
	Swiftly		
	Grip rail	size 1, Item code: 6833	

The reception tubes must be prepared before the grip rail is mounted:

1. With a flat object, carefully remove and check under the plastic casing.
2. Screw in the locking pins. Finish if necessary by tightening with a 10 tool.

Insert the grip rail in the tubes. The height can be set by using the locking pins. Turn over the grip rail to vary the distance to the child.

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(GB) Rain shield for sun and rain cover, transparent

Item code: 6739

The combination of sun and rain cover and rain shield offers optimum protection against rain. Simply pull the rain shield over the sun and rain cover. In addition, the rain shield can be attached right and left at the push bar with velcro.

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(GB) **Rain cape** size 1, Item code: 6340 size 2, Item code: 6540

Pull the rain cape over backrest, seat, and footrest, and pull it in shape. Fix the straps in the foot area to the chassis.

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Ⓞ Sun and rain cover with leg blanket Item code: 6637
Ⓞ Sun and rain cover with leg blanket, transparent Item code: 6638

Mount the enclosed canopy receivers (plastic) with the provided screws to the left and right of the frame:

1. Detach the screws on the backrest bow. To mount the plastic receivers release the holes in the frame by removing the plastic plugs.
2. Now mount the receivers to the outside of the frame according to the drawing (straight sides of the receivers face upwards) and insert the plastic clamps of the canopy into the receivers.

To attach the leg blanket button it to the canopy.

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Soft pelvic harness

Item code: 6614

Lay the soft pelvic harness onto the seat surface (buckles face downwards). Guide both belts running backwards through the upholstery and fix them at the rear of the seat surface (Fig. 1). The two **bottom belt ends** can be attached with cam buckles to the back plate (Fig. 2). Guide the belts left and right next to the back upholstery to the corresponding cam buckle and fix them. Close the back upholstery.

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Ⓞ	H-belt	Item code: 6629		
	Butterfly / chest harness			
		Item code: 6616 (S)	Item code: 6617 (M)	Item code: 6717 (L)
	Seat waistcoat			
		Item code: 6631 (S)	Item code: 6632 (M)	Item code: 6731 (L)

Cut the backrest padding in the quilted seam depending on the desired mounting height. Pass the two **upper belts** through the holes in the upholstery and insert them into the back plate. (Fig. 1).

Open the velcro of the back upholstery. Insert the belts into the cam buckles and adjust the desired belt length (Fig. 2).

The two **bottom belt ends** can be attached with cam buckles to the back plate (Fig. 3). Guide the belts left and right next to the back upholstery to the corresponding cam buckle and fix them. Close the back upholstery.

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Basket

Item code: 6813

Fix the basket at the four provided loops underneath the seat surface. Make sure the wider cut side of the basket faces rearwards.

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Head pillow

Item code: 6824

Place the head pillow at desired height. Guide both Velcro straps around the back and Velcro-fasten them.

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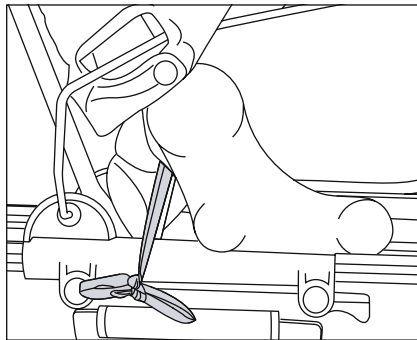
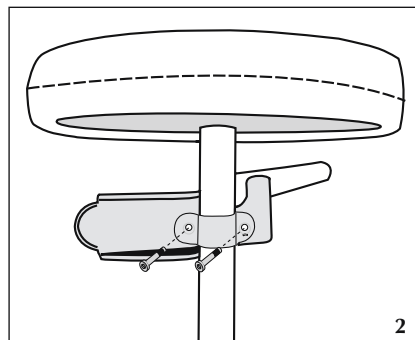
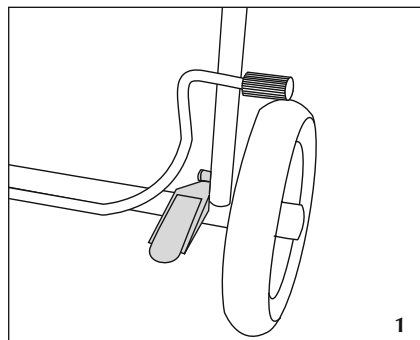
GB

Sleeping bag

size 1, Item code: 6619 (padded)
size 1, Item code: 6618 (woven fur)

size 2, Item code: 6719 (padded)
size 2, Item code: 6718 (woven fur)

Lay the body contoured sleeping bag on the seat surface. Guide the strings around the black plastic parts at the pivot point between the seat surface and the backrest tube and bind them together. Fix the clamping buttons in the foot area to the rear tube of the footrest.

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GB Tip-assist

Item code: 6829

To mount the tip-assist lay the rehab stroller on the side (with the right side facing upwards - viewed from the back).

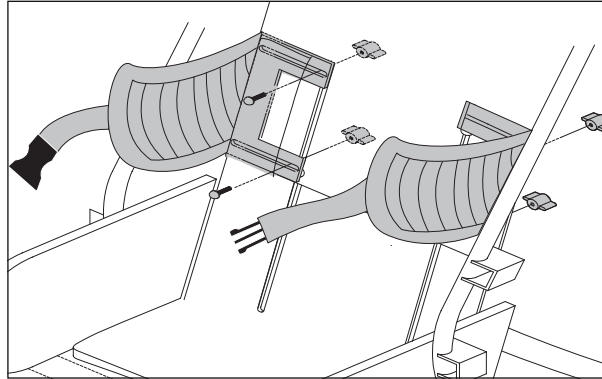
Position the tip-assist at the rear axle directly at the tube next to the right rear wheel. Now fix the tip-assist to the counterpiece at the tube using the provided allen screws and washers. (Fig. 2 shows bottom view)

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GB Lateral trunk supports, flexible with chest belt

Item code: 6633

Open the back padding in the quilted area at the measured thorax height. Insert both button-headed screws through the grooves of the lateral trunk support, guide the lateral trunk support through the back and attach them to the backrest's longitudinal slots using the hand wing screws. Washers can also be used.

You can adapt the supports individually by carefully bending them. Close the chest belt with the buckle. The belt length is individually adjustable on both sides.

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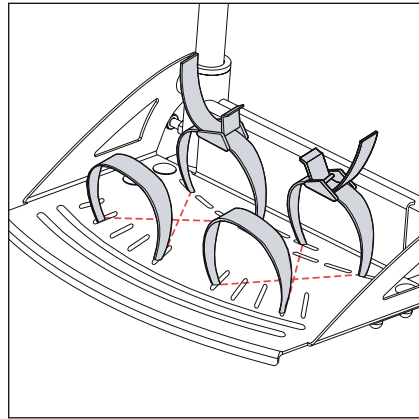
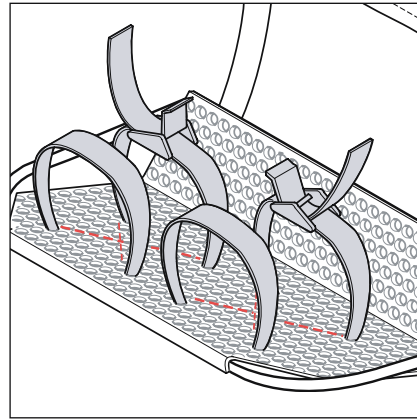
Foot straps

Item code: 6648

Select the hole spacing according to foot size and thread the belt through the holes in the footrest.

The belt crosses underneath the footrest.

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(GB) Abduction block size 1, Item code: 6620 size 2, Item code: 6720

Insert the abduction block through the seat upholstery into the groove in the seat plate as far as it will go. Make sure the narrow end of the block is facing to the child's body. Ensure that the steel rails from the abduction block are inserted through the upper and lower loophole of the entry. Insert the screw from the front of the seat and screw it tight to secure the abduction block.

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