



Tendercare Pushchair Clinical Guide

This is an interactive clinical guide for our range of pushchairs. It has been designed to enable easy comparison within the range and to help pick the best fit for your client. If there is any information not in the guide which you would like added, send us an email and we can add this for you.

Clinical Information

- Seat & Chassis
- Compatibility Chart
- Pushchair Comparison Chart
- Pushchair Sizing Chart

Pushchair Overview & Key Features

- Cloud
- Snappi Cloud
- Snappi
- Snappi Swirl
- Snazzi
- Swifty

Contents

Seat & Chassis Compatibility Chart

Tendercare seats and chassis are designed to be interchangeable. This chart details which seats can fit onto which chassis. Please note, the seat and chassis must be the same size i.e. a size 1 seat can only fit on a size 1 chassis, likewise with a size 2.

Seat / Chassis Compatibility	Snappi Chassis	Cloud Chassis	Swirl Chassis
Snappi Seat	✓	✓	✓
Cloud Seat	✗	✓	✗

* Please note, the Cloud pushchair uses the Cloud interface and so can only fit onto Cloud chassis with the Cloud interface. The Snappi Cloud uses the Snappi interface so can only fit onto Cloud chassis with the Snappi interface



Pushchair Comparison Chart	Cloud	Snappi	Snappi Cloud	Snappi Swirl	Snazzi	Swiftly
Tilt in Space	✓	✓	✓	✓	✓	✗
Back Recline	✓	✓	✓	✓	✗ (not full recline but back opens to 90 or 95)	✓
Level of Support	High	High	High	High	Medium	Medium
External Headrest Capability	✓	✗	✗	✗	✗	✗
Medical Equipment	Equipment tray (cloud chassis fits tall equipment) 1 or 2 vertical o2 cylinders to side mount o2 1 or 2 hook and brackets to enable excess equipment to be hung	Equipment tray with a o2 section or without. 1 or 2 vertical o2 cylinders to side mount o2	Equipment tray (cloud chassis fits tall equipment) 1 or 2 vertical o2 cylinders to side mount o2 1 or 2 hook and brackets to enable excess equipment to be hung	Equipment tray (for short items)	Equipment tray with a o2 section or without.	Shopping basket O2 cylinder
Weight Capacity	55kg	40kg	55kg	40kg	40kg	35kg
Pushchair Weight	23.7kg (seat and chassis combined)	7.5kg seat and 11.5kg chassis (size 1) 9.5kg seat and 12.5kg chassis (size 2)	7.5kg seat and 11.2kg chassis (size 1) 9.5kg seat and 12.2kg chassis (size 2)	7.5kg seat and 13.4kg chassis (size 1)	16kg (size 1) 18kg (size 2)	12.4kg (Swiftly) 18.2kg (Swiftly 2)
All in One or 2 Part Fold	2 part fold	2 part fold	2 part fold	2 part fold	All in one fold	All in one fold
Forward / Rearward	Forward and Rearward (family can change orientation)	Forward or Rearward (WCS can change orientation)	Forward or Rearward (WCS can change orientation)	Forward and Rearward (family can change orientation)	Forward only	Forward only
Crash Tested	✓ (when forward facing)	✓ (when forward facing)	✓ (when forward facing)	✓ (when forward facing)	✓	✓
Postural Accessories / Customisation	Full range of postural accessories (inc headrests, harnesses, pads, laterals etc) Custom accessories (e.g. pressure relief covers)	Full range of postural accessories (inc headrests, harnesses, pads, laterals etc) Custom accessories (e.g. pressure relief covers) Customisation to seat (e.g. longer hip guides, deeper side wings etc.)	Full range of postural accessories (inc headrests, harnesses, pads, laterals etc.) Custom accessories (e.g. pressure relief covers) Customisation to seat (e.g. longer hip guides, deeper side wings etc.)	Full range of postural accessories (inc headrests, harnesses, pads, laterals etc.) Custom accessories (e.g. pressure relief covers) Customisation to seat (e.g. longer hip guides, deeper side wings etc.)	Full range of postural accessories (inc headrests, harnesses, pads, laterals etc.) Custom accessories (e.g. pressure relief covers) Some customisation to seat	Full range of postural accessories (inc headrests, harnesses, pads, laterals etc.) No customisation to seat

Pushchair Sizing Chart

Pushchair Sizing Chart	Cloud (size 1)	Snappi Cloud (minimiser, size 1, size 2)			Snappi (minimiser, size 1, size 2)			Snappi Swirl (minimiser and size 1)		Snazzi (size 1 and size 2)		Swiftly (Swiftly and Swiftly 2)	
Seat Depth	190 - 290mm	140 - 310mm	195 - 310mm	280 - 390mm	140 - 310mm	195 - 310mm	280 - 390mm	140 - 310mm	195 - 310mm	210 - 290mm	270 - 350mm	220 - 285mm	355 - 415mm (can be further reduced to 305 or 330mm by fitting pads)
Seat Width	180 - 290mm	170 - 290mm	170 - 290mm	250 - 350mm	170 - 290mm	170 - 290mm	250 - 350mm	170 - 290mm	170 - 290mm	195 - 285mm	225 - 335mm	190 - 340mm (adjustable by pads)	220 - 395mm (adjustable by pads)
Back Height	Without external headrest: 470 - 630mm With external headrest: 450 (with 3” seat pad fitted) - 630mm	475 - 675mm	500 - 675mm	590 - 760mm	475 - 675mm	500 - 675mm	590 - 760mm	475 - 675mm	500 - 675mm	600mm Fixed	670mm Fixed	620mm Fixed	220 - 395mm (adjustable by pads)
Lower Leg Length	195 - 300mm	120 - 280mm	150 - 280mm	215 - 360mm	120 - 280mm	150 - 280mm	215 - 360mm	120 - 280mm	150 - 280mm	120 - 330mm	120 - 330mm	160 - 330mm	210 - 370mm

* If measurements are needed outside these ranges, please contact Tendercare to discuss e.g. if you need to reduce a size 2 Snappi by 2”/50mm, we could make you a back pad or if you would like to have a shorter lower leg drop, we could make a footpad.. The Snappi range (i.e. Snappi, Snappi Cloud and Snappi Swirl) and Cloud pushchair have the most capability for custom measurements. If you let us know your measurements, we can let you know the best way / product to achieve this.

Cloud Pushchair



Colour Options



* Black spacer fabric with two tone black with grey, red, lime, blue or magenta sides. Bespoke covers and materials available. Give us a call to discuss your bespoke requirements.

Pushchair Features



Innovative new style of interface which enables the seat to fit into the chassis in both forward and rearward.



Compatible with a range of external headrests.

Back recline, without fixed positions, of 90 - 140°. Dynamic option available.

Tilt in space, without fixed positions, of 10-40°.

Height and angle adjustable push handles.

Highly adjustable in order to ensure both maximum growth and ideal seating position.

To facilitate quicker and smoother clinic appointments, the seat depth, seat width and angle of the hip guides are handwheel adjustable.

Hip guides in addition to being angle, width and length adjustable are also height adjustable and fitted with a curved top so can be used as armrests.

Tip assist.

Adjustable knee angle.

Soft-touch, easy-to-operate, pin-lock brake. Hub brake option also available.

Height and angle adjustable flip-up footrest.

Large amount of space underneath the seat for taller equipment, like suction machines, to fit standing up.

Front and rear transit brackets, fitted as standard.

Swivelling front castors. Front locking castors option available.

Crash-tested (forward facing position only) according to ISO 7176-19.

Exceptionally high weight capacity of 55kg.

Click for
“Pushchair
Sizing Chart”
on page 8

Snappi Cloud



Part of the
Snappi range see
“Seat & Chassis
Compatibility Chart”
on page 4

Colour Options



* Black spacer fabric with two tone black with grey, red, lime, blue, magenta, red piping or pink piping sides. Bespoke covers and materials available. Give us a call to discuss your bespoke requirements.

Pushchair Features

Back recline, without fixed positions, of 90 - 140°. Dynamic option available.

Available in forward or rearward facing.

The Snappi cloud uses the proven and ever-popular Snappi seat ensuring you get all the features and support of the Snappi seat in a stylish, new chassis.

Tilt in space, without fixed positions, of 0-40° (size 1) & 0-45° (size 2).

Height and angle adjustable push handles.

Highly adjustable in order to ensure both maximum growth and ideal seating position.

Cutting-edge, oval-tube design.

Adjustable knee angle.

Soft-touch, easy-to-operate, pin-lock brake. Hub brake option also available.

Height and angle adjustable flip-up footrest. Dynamic option available.

Exceptionally high weight capacity of 55kg.

Front and rear transit brackets, fitted as standard.

Swivelling front castors. Front locking castors option available.

Crash-tested (forward facing position only) according to ISO7176/19.

Click for
“Pushchair
Sizing Chart”
on page 8

Snappi



Part of the Snappi range see “Seat & Chassis Compatibility Chart” on page 4

Colour Options



* Black spacer fabric with two tone black with grey, red, lime, blue, magenta, red piping or pink piping sides. Bespoke covers and materials available. Give us a call to discuss your bespoke requirements.

Pushchair Features

Back recline, without fixed positions, of 90 - 135°.

Available in forward or rearward/parent facing.

Tilt in space, without fixed positions, of 0-40°.

Soft-grip, height and angle adjustable push handles.

Highly adjustable in order to ensure both maximum growth and ideal seating position.

Compact and easy-to-fold.

Adjustable knee angle.

Soft-touch, easy-to-operate, pin-lock brake. Hub brake also available.

Front and rear transit brackets, fitted as standard.

Swivelling front castors (with front locking option).

Large, woven fabric shopping basket as standard.

Crash-tested (forward facing position only) according to ISO7176/19.

Height and angle adjustable flip-up footrest.

Click for “Pushchair Sizing Chart” on page 8

Snappi Swirl

Part of the
Snappi range see
“Seat & Chassis
Compatibility Chart”
on page 4



Colour Options



* Black spacer fabric with two tone black with grey, red, lime, blue, magenta, red piping or pink piping sides. Bespoke covers and materials available. Give us a call to discuss your bespoke requirements.

Pushchair Features

Back recline, without fixed positions, of 90 - 135°. Dynamic option available.

Tilt in space, without fixed positions, of 0-40°.

Highly adjustable in order to ensure both maximum growth and ideal seating position.

Front and rear transit brackets, fitted as standard.

Swivelling front castors with locking capability as standard.

Large, woven fabric shopping basket as standard.

Compact and easy-to-fold.

Buggy rotates 360° with the child still in the seat.

Compatible with multiple Tendercare seating systems including our Snappi, Snugseat and Modular seats.

Height and angle adjustable push handles.

Frame protector down the sides of the push handle.

Adjustable knee angle.

Height and angle adjustable flip-up footrest. Dynamic option available.

Soft-touch, easy-to-operate, pin-lock brake. Hub brake option also available.

Crash-tested (forward facing position only) according to ISO7176/19.

Click for
“Pushchair
Sizing Chart”
on page 8

Snazzi



Colour Options



* Black spacer fabric with two tone black with grey, red, blue, magenta or red piping sides. Bespoke covers and materials available. Give us a call to discuss your bespoke requirements.

Pushchair Features

All-in-one, folding buggy.

Height-adjustable, soft-grip push handles.

Fixed back recline of 90° or 95°.

Removable seat and back upholstery; made from a breathable fabric; and available in five colour options.

Easy-to-use, soft-touch pin-lock brake.

Substantial space underneath the pushchair for toys, medical equipment and shopping.

Woven fabric shopping basket as standard.

Tilt in space, without fixed positions, of 10-40°. The tilt mechanism is easy to operate and requires just one hand to adjust.

Highly adjustable in order to ensure both maximum growth and ideal seating position.

Transit tie down system (fitted) which enables Snazzi pushchairs to be transported in wheel-chair-adapted vehicles.

Swivelling front castors for effortless manoeuvrability.

Depth and height adjustable footrest.

Crash tested in accordance with ISO 7176/19.

Click for
“Pushchair
Sizing Chart”
on page 8

Swiftly & Swiftly 2



Fabric Colour



* Black spacer fabric with red piping

Pushchair Features

Lightweight, aluminium, fold-in-one pushchair.

Integrated seat unit.

Adjustable hip angle.

Adjustable seat depth offering a large range of growth of up to 65mm.

Height adjustable, flip-up footrest.

Crash-tested according to ISO 7176/19 and ANSI/RESNA WC/vol.1 section 19 6/22/99 draft.

Front and rear transit brackets, fitted as standard.

Back recline, without fixed positions.

Fixed angle of tilt in space.

Height and angle adjustable, soft-touch push handles.

5 Point harness.

Easy-to-operate foot brake. Hub brake option also available.

Woven fabric shopping basket as standard.

Swivelling front castors allowing for effortless manoeuvrability. Front locking option is available.



Click for
“Pushchair
Sizing Chart”
on page 8