Modular Seating System

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Please contact your local, authorised distributor if you have any questions regarding the use, maintenance or safety of your pushchair. In case there is no authorised distributor in your area or you have any questions, kindly contact:

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Tendercare Ltd.

Unit 10 Minster Court, Courtwick Lane, Littlehampton, BN17 7RN

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Email: info@tendercareltd.com

Web: www.tendercareltd.com

If there is a serious and unexpected accident with the device, please immediately inform both Tendercare Ltd and the responsible authorities. The quality management system of Tendercare Ltd is certified to ISO 9001:2015 and ISO 13485:2016.

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2. INTRODUCTION

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The Modular Seating system is a highly adjustable seating system designed to accommodate a wide variety of seating needs. The Modular Seating system is a Class 1 medical device designed for regular use. The device can be used by children and adults with disabilities who lack the ability to maintain a seated position. The target group for the device is based on functional ability and not a specific health condition or age. This Modular seating system is designed to be used as a mobility aid and not as a static seating system; as such, the occupant should not be left in the chair without an attendant holding the push handles of the wheelchair/pushchair base. The product can be used at a walking pace both indoors and outdoors. The Modular seating system can be used for transport in a wheelchair adapted vehicle in accordance with ISO 7176-19 (please see section 13 'Transporting the Wheelbase' for full instructions and details of specific "crash tested bases"). A wide range of accessories are available for the Modular Seating System to enable customisation to an individual's specific requirements.

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Risks Associated with Use and Contraindications

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If used properly, risks can be eliminated during use. There are no known contraindications. The occupant's safety is your responsibility. As such we recommend that you read the complete user manual prior to using your Modular seating system.

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This Manual is designed as a general guide to explain the use and functioning of the Tendercare Modular Seating System.

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With care and regular maintenance your seating system will provide you with many years of good service. If, after you have used the seating system for a few weeks, you feel that it is not meeting your needs, you are advised to discuss the matter with your prescriber or supplier

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The Tendercare Ltd Modular Seating System is designed for a wide range of people who will benefit from a comfortable supportive seating system. The system is supplied in three standard sizes, 1, 2 & 3. There is also individual custom-made version that is intended only for use by the assessed client.

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2. UNPACKING

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Depending on the purchase arrangements the seating system may be supplied pre assembled on a wheelchair or supplied in a cardboard carton. The size and weight of the carton will depend upon the options ordered.

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Warning: - The transit carton may be heavy so moving and unpacking must be done with care. Observe all lifting and handling regulations.

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Stand the Carton upright making sure it is supported and cannot fall over. Open the carton and remove the contents

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Check that you have the following items: -

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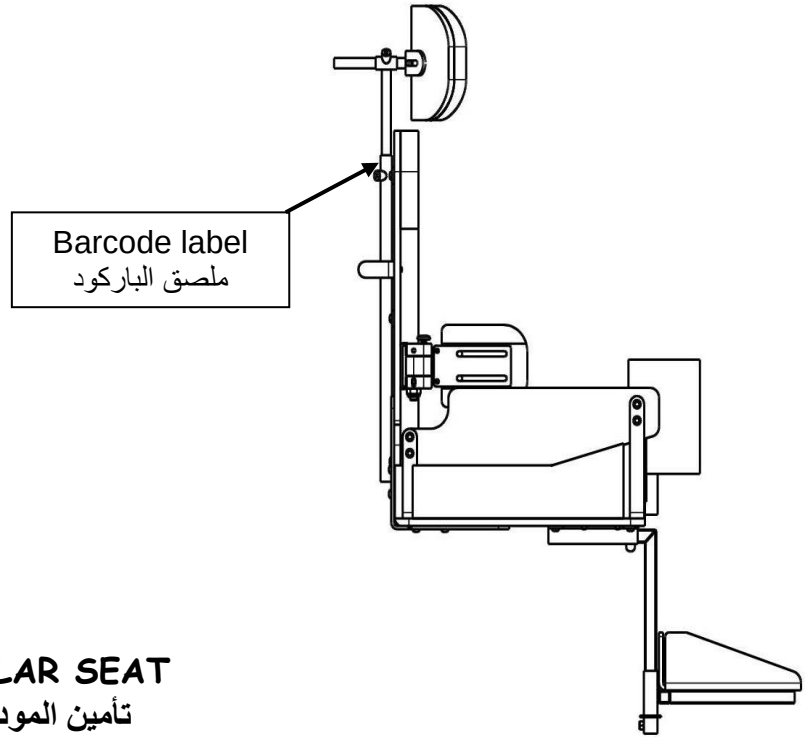
If any items are damaged or missing please contact your supplier or Tendercare Ltd.
Preferably by fax 01903 734083 within 24 hours of delivery.

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After unpacking and checking you have all components and that they are in good condition, carefully dispose of the packaging at your local recycling centre.

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The modular seat is often supplied pre-fitted to

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a range of different bases. The following are details of some of the most common combinations. For any wheelbase not listed below, please refer to the manufacturer's guidelines or contact Tendercare Ltd directly

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4.1 Wheelchairs with Flexible Seat & Back Upholstery

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Remove armrests from host wheelchair to improve access. (See Fig 1 opposite)

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Place Modular Seat onto wheelchair upholstery. Make sure that the two locators, one each side of the modular seat backrest are located on the

outside of the wheelchair push handles. (See Fig 2 below)

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Ensure that the two black retaining straps are not tangled or obstructed. Pass the straps round the wheelchair backrest

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Push both ends of the two buckles together to lock the catch. Tighten the straps by pulling the free end after you have locked the buckle. (See Fig 1 above)

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To loosen the straps press the red button to release the buckle. Holding the end with the tongue turn it at a right angle to the strap (see picture opposite). The strap will then slide through the buckle

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When the seat is secure refit the armrests

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To remove the seat from the wheelchair, release the black belts by pressing the red button on each catch, then lift the seat off the wheelchair.

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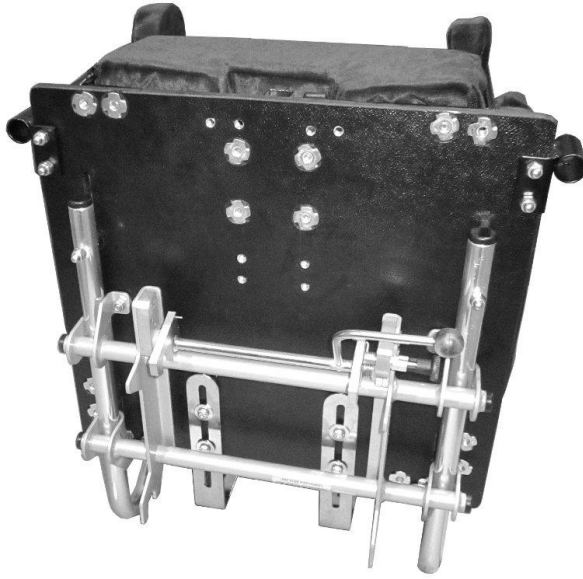
IMPORTANT: - Make sure the black retaining straps when fastened together have no slack in them and that the buckles are properly locked. Check this every time the seat is used

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The Snappi Interface is supplied pre-fitted to the underside of the modular as shown (left).

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To fit to the pushchair, hook the rear of the interface into the locating points on the wheelbase and rotate forward until the front clips of the interface latch onto the locking points on the wheelbase

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For full details of working with the Snappi Wheelbase and Interface, please refer to the dedicated user manual (which will be included in the box with the wheelbase, and is also available to download from the Tendercare website)

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5. POSITIONING THE CLIENT INTO THE MODULAR SEAT

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Note: - The client must be positioned correctly into the Modular seat at all times for it to work properly. If you are unsure what the correct position is, of either the occupant or of any accessory, you must contact the prescribing clinician/therapist immediately. You must not use the Modular Seating System until you have been shown correct usage and are confident in its application.

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The headrest must be adjusted into the correct position. See sections 6.3 and 6.4

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The Butterfly (Chest) harness must be kept low over the user's chest away from the neckline to avoid serious injury. To fit and adjust the harness, see section 6.5.1

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The pelvis must be positioned as far back as possible within the Seat. The pelvic strap should be positioned as low as possible over the hips and **not** over the stomach. See section 6.5. 2

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Position the feet correctly onto the footrest/support – fit the toe and ankle straps (if fitted). See Section 6.6

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Fit and adjust the Pommel or Knee Blocks depending upon which item is being used. See sections 6.7 Pommel & 6.8 Knee Blocks.

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Adjust the Hip and Thoracic pads so that they maintain the desired posture of the user. See sections 6.9 Thoracic Pads & 6.10 Hip Pads.

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Because of the nature of Modular Seating, the user should be given time to become accustomed to the new seat position, so short periods of seating should be introduced and built up over a period of time.

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The seating system has a wide range of adjustment so that it will meet the needs of large number of people. This section describes how to make the adjustments

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The only tools required are a 4 mm “Allen Key” for the seat and a 5 mm “Allen Key” for the headrest. A flat blade screwdriver, 10 mm and 13 mm spanners are required for the knee block

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The seat depth can be adjusted in 10 mm steps. To adjust the seat depth (See pictures opposite and below). Remove the four screws from both brackets / hinges. Move the brackets / hinges to the desired position and refit the screws

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The backrest height can be adjusted in 10 mm steps. To adjust the backrest height, (See pictures opposite and below) remove two screws from each bracket / hinge. Move the backrest to the desired position and refit the screws.

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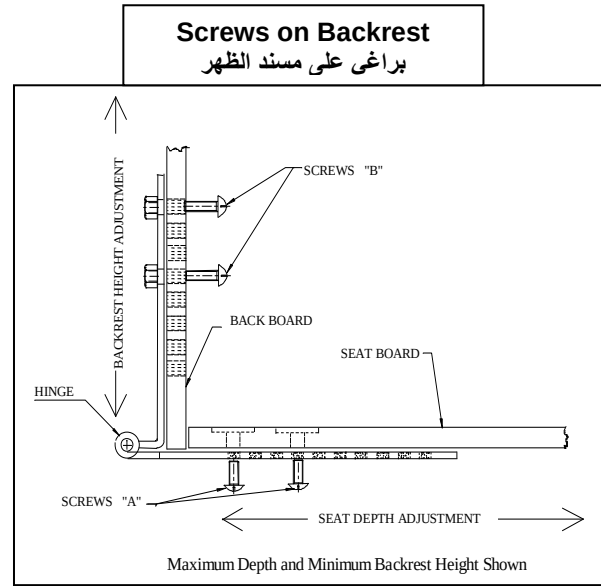
Important

Make sure that all screws are tight before using the seat.

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

To adjust the backrest height, loosen the 4 x M6 bolts using a 4mm hexagon key. Slide the backrest to the desired position, and then tighten the 4 x M6 bolts again to lock it in position.

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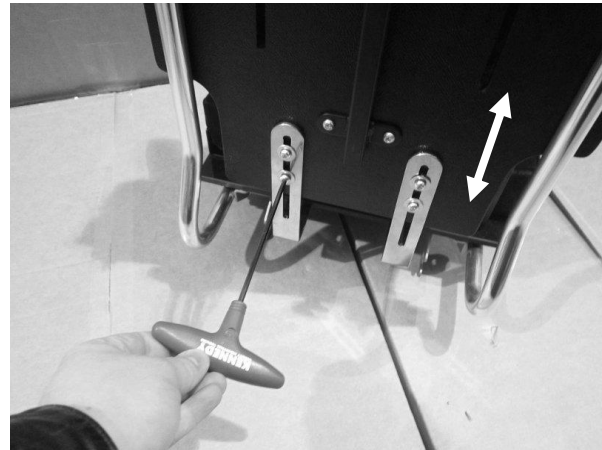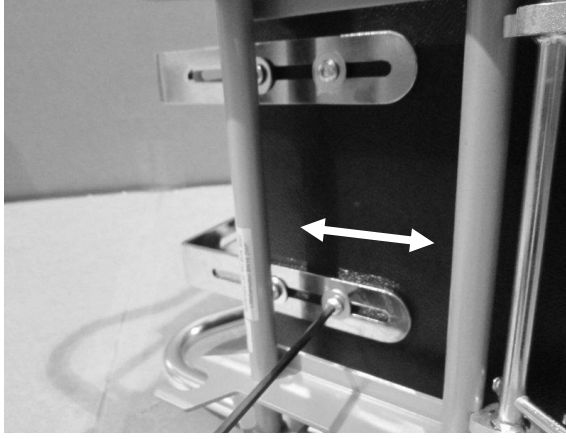Place the modular on its side on a flat surface to provide access to the underside of the seat.

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Loosen the 4 x M6 bolts using a 4mm hexagon key. Slide the seat base to the desired position, and then tighten the 4 x M6 bolts to lock it in position

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6.2 Cushions

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The seat is supplied with two back cushions one sacral support and one seat cushion. To fix in position, simply press the cushions onto their Hook & loop Mountings.

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NOTE: - There are two styles of seat cushion, one has a cut out at the front to accept the pommel the other has a plain front for use with knee blocks

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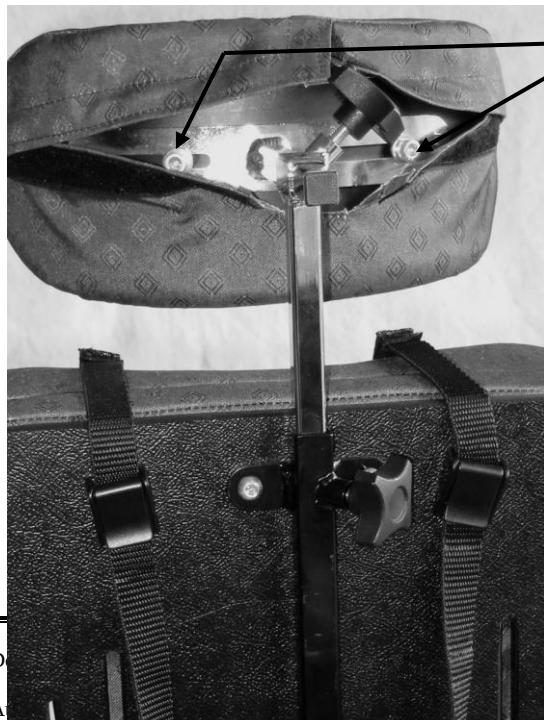
The headrest can be adjusted laterally (side to side), horizontally (forward & back) and vertically (up & down).

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To adjust the headrest side to side, pull the Hook & loop seal on the cover apart and loosen

the two screws using a 5 mm "Allen Key" by turning the screws anti-clockwise. (See picture opposite). Move the headrest to the desired position and tighten the screws by turning them clockwise. Refit the cover by pressing the Hook & loop strips together

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To move the headrest forward and back,

turn the thumb wheel anticlockwise (See *picture opposite*). Move the headrest to the desired position and tighten thumb wheel by turning it clockwise.

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To adjust the height of the headrest, turn the thumb wheel (See *picture opposite*) anticlockwise. Move the headrest to the desired height and tighten the thumb wheel by turning it clockwise

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IMPORTANT: - The headrest should remain in position whilst the user is being transported in a vehicle.

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The ball joint headrest has all the adjustments offered by the standard headrest, and also allows for angle adjustment thanks to the ball joint mounting (see right).

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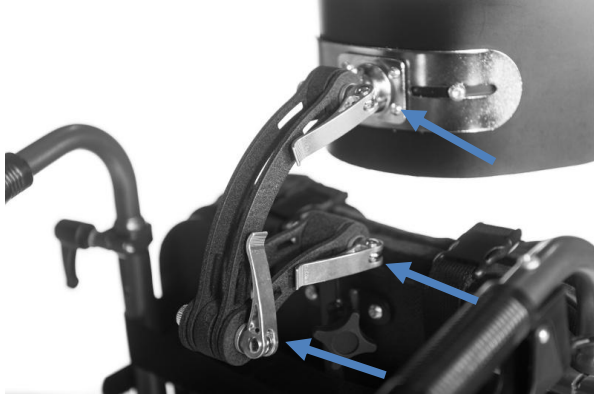
To adjust the angle of the multi-positional headrest, loosen (do not remove) the 4 clamp screws on the ball joint bracket, rotate the headrest as necessary and then lock by tightening the 4 screws

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The new style multi-positional headrest has the same ball joint as the original (for details on how to adjust the ball joint please refer to section 6.4.1, above). This is coupled with a versatile articulated arm that provides a wider range of adjustment for the position of the pad

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To adjust the position of the articulated arm, pull back the quick lock lever so that the joints are loose. Move the arm into the desired position, and then lock in place by pushing the quick lock levers back against the arm.

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The harness system comprises a padded butterfly harness that fits over the neck and a padded lap strap. The harness is designed to provide comfortable support for the user.

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The harness comprises a padded section that sits against the user's chest, two shoulder and two bottom straps. The shoulder straps are locked in a position to suit the user and final adjustment is made on the bottom straps. (See picture opposite).

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IMPORTANT: - In order to ensure that the Butterfly (Chest) harness can not rise up and cause harm to the user, every time the harness is in use, you must ensure that all cam buckles/fixings are properly closed and that the harness is correctly positioned a minimum of 75mm below the user's neckline. Failure to comply with this could result in serious injury.

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To adjust the shoulder straps. Pass the straps over the user's shoulder. Lift up the harness lock release levers and pull the straps to the desired tension. Lock in place by pressing down the release lever. (see picture opposite).

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To lock the bottom two catches simply push the two halves together until they click

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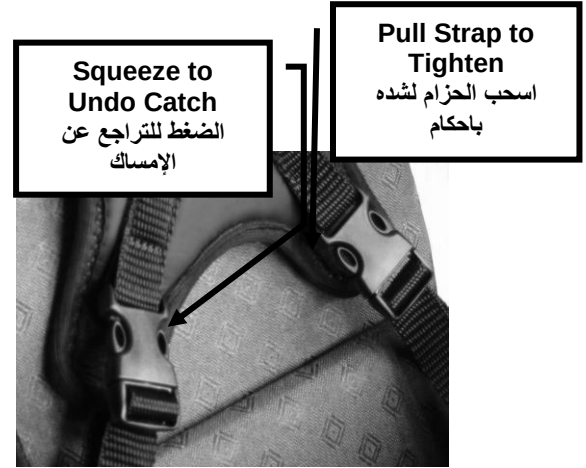
To tighten the strap, pull on the loose end. To loosen the strap, twist the catch so that it is at right angles to the strap and pull.

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To undo the catch, press on each side of the buckle and pull the two halves apart

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The pelvic strap comprises a padded section that sits on the user's lap and an adjustable strap that is anchored either side of the seat

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To fasten the lap strap, clip the 2 halves of the buckle together until the click, then tightens by pulling the ring pull (See photograph opposite)

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To release, press on each side of the buckle and pull the two halves apart.

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IMPORTANT: - The lap strap must be located as low down over the hips

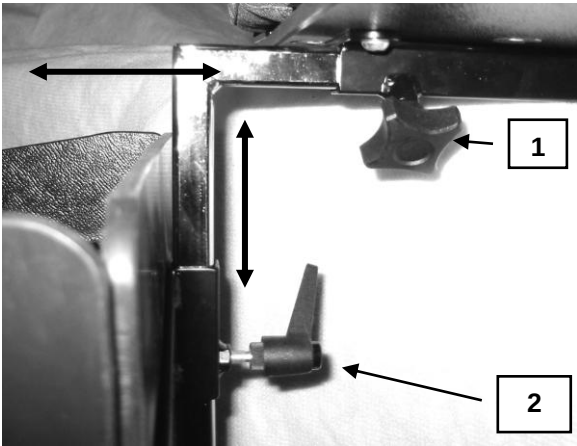
as possible. It must not cross the stomach of the passenger.

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IMPORTANT: - The harnesses supplied with the seat are NOT designed for use as a vehicle harness. See transport section 10 below, for advice on travelling in a vehicle

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To fit the footrest, turn hand-wheel “1” anticlockwise (see picture opposite) and insert the footrest vertical tube into the mounting tube. Tighten the hand-wheel by turning it clockwise until the support bar is locked in place.

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Adjust the footrest height by turning hand wheel “2” at the back of the footrest

anticlockwise. (See picture opposite). Move the footrest to the desired height and retighten the hand wheel by turning it clockwise

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If toe and ankle straps are fitted, simply place them around each foot and fasten using the Hook & loop fastening or plastic “CAM” buckles depending upon which type is fitted

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Footrest must not be used to assist transfer of the patient into the seat. Patient must not stand on the footrest.

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The new style footrest is available on Size 1 and 2 modular systems and provides a more attractive foot tray.

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The adjustments function the same as the old style footrest (please see section 6.6.1 above)

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The only difference is the ankle straps are fitted to the back of the foot tray rather than the base (see arrow in photo, left)

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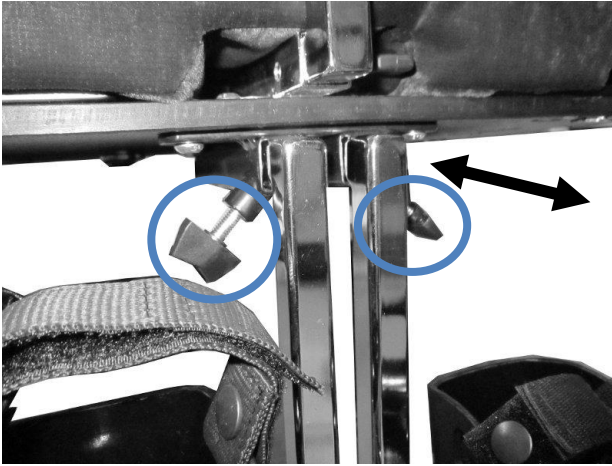
The modular can be supplied with an optional independent footrest kit, which provides exceptional flexibility and adjustment for foot positioning.

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To adjust the depth of the footrest, loosen the 2 thumbscrews on the mounting as shown (see circles, left)

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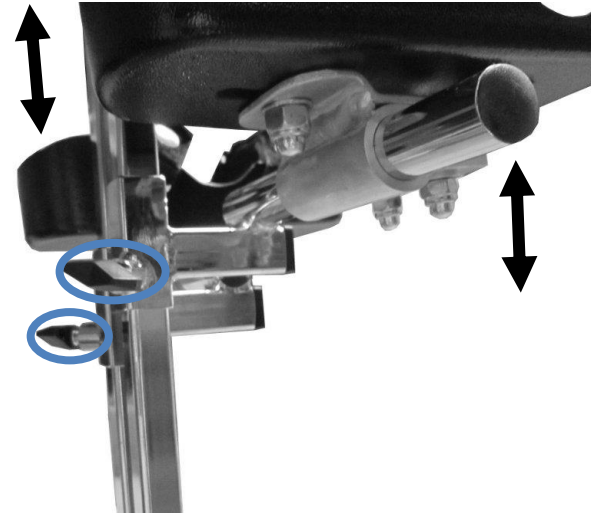
The height of the 2 footplates can be adjusted independently

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To adjust, loosen the thumbscrew (circled), slide the footplate to the desired position then tighten the thumbscrew again

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
The depth of the footplates can also be independently adjusted.

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To change the footplate position, loosen the 2 x 6mm nuts on the underside of the plate (circled) using a 10mm socket or spanner. Slide the foot plate to the desired position and tighten

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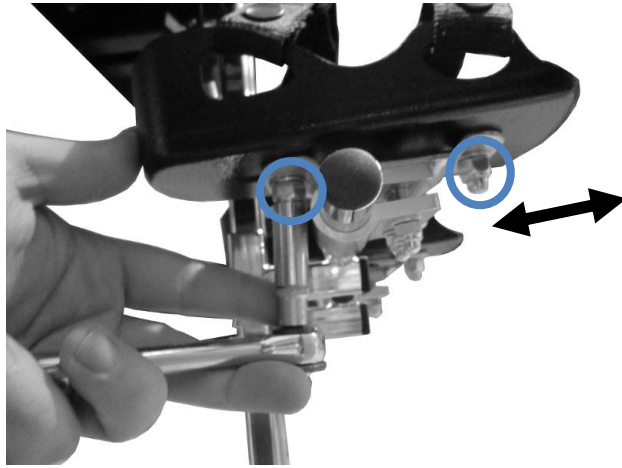The angle and width of the footplates can be adjusted

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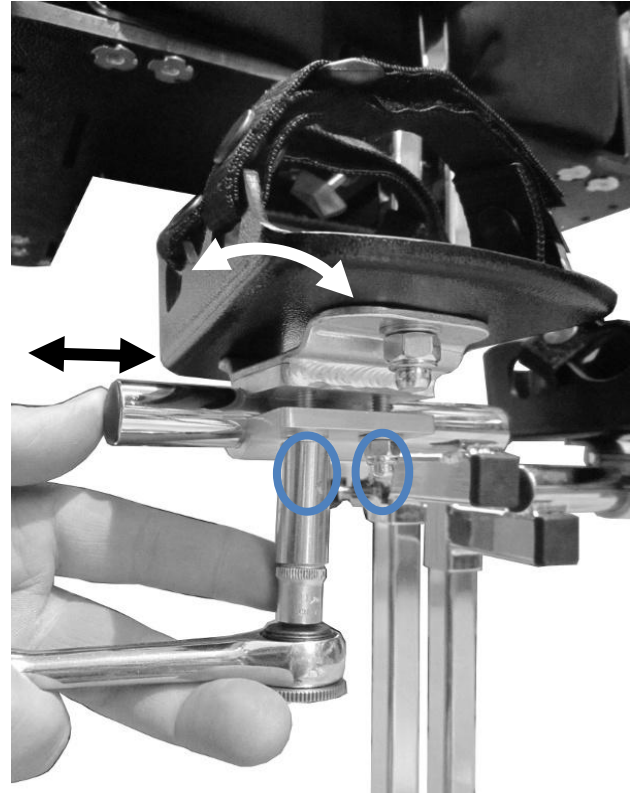
To adjust, loosen the 2 x M6 nuts on the underside of the mounting clamp (circled) using a 10mm socket or spanner

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Move the foot plate into the desired position, and then tighten the nuts again to lock it in place.

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To fit the pommel, lift the front of the seat cushion then lift spring loaded release catch a quarter turn and insert the pommel support into the pommel mounting tube. Turn the spring release until the support bar is locked in place. (See picture opposite)

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The distance between the backrest and Abduction

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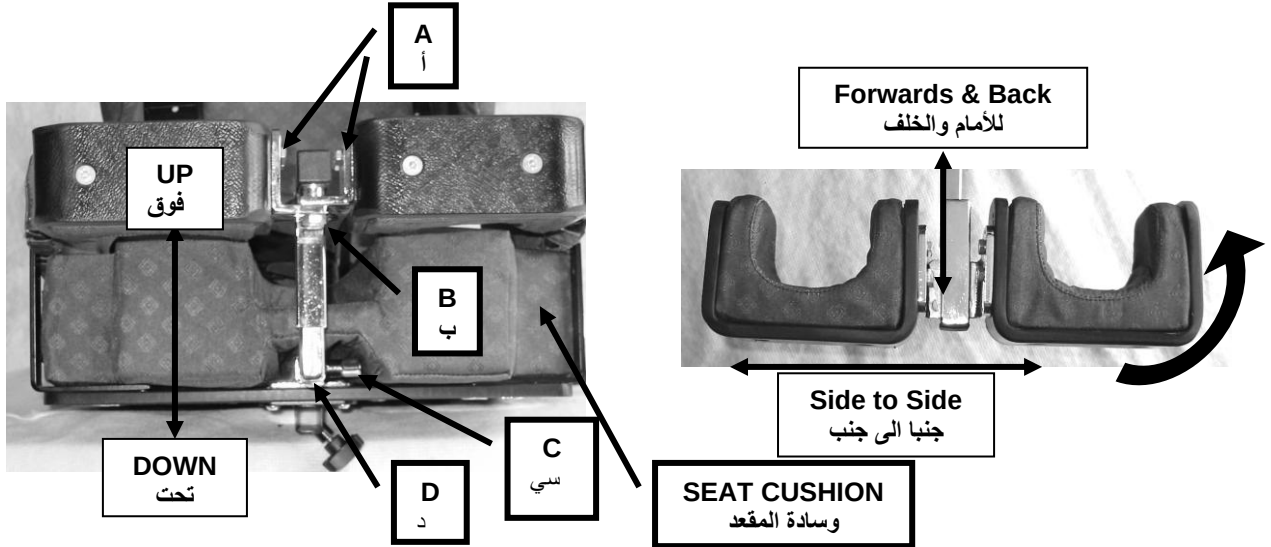
To remove the pommel move the release catch a quarter turn then pull the pommel from the seat.

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6.8 Knee Blocks

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The knee block assembly has a wide range of adjustment, vertically and horizontally (forward and back). Also, each knee block can be moved independently, side-to-side and angled to suit the user's posture

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To fit the knee block assembly, lift the spring-loaded release catch “C” (see above picture) a quarter turn and insert the knee block support into the mounting tube. Turn the spring release until the support bar is locked in place

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The distance between the backrest and knee block can be adjusted in 20 mm steps. To adjust, move the release catch a quarter turn then slide the knee block to the desired position. Make sure that the release catch locks the knee block.

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To remove the knee block move the release catch a quarter turn then pull the assembly from the seat.

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To adjust the height, loosen screw “D” using a Flat Blade screwdriver. (See picture above). Move the knee block to the desired height and tighten the screw.

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To adjust the angle and width of the knee block loosen two screws “B” using a 13 mm spanner. (See picture above). Move the knee block to the desired position and retighten the screws.

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To adjust the forward / back movement of each knee block loosen screw “A” using a 10mm spanner (See picture above). Move the knee blocks to the desired position and retighten the screws.

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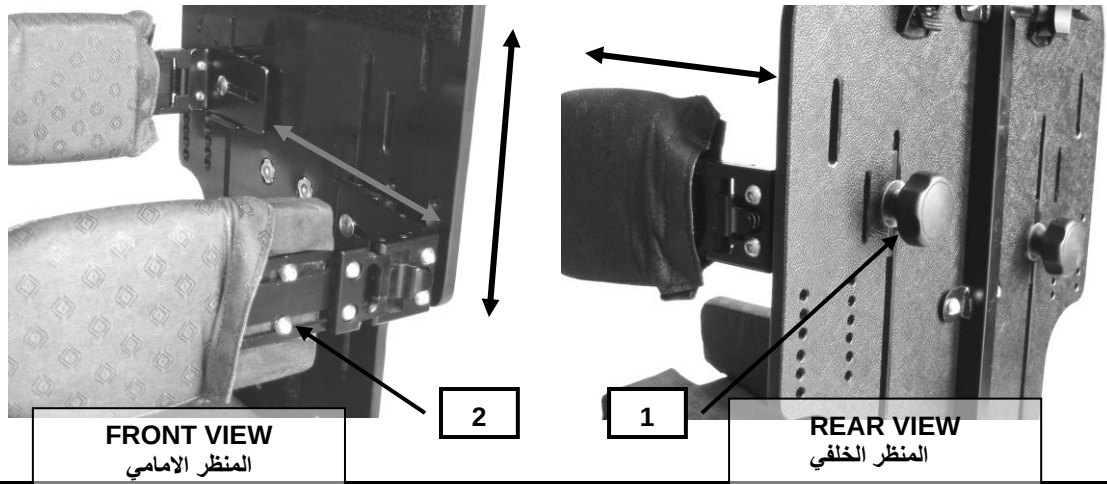
6.9 Thoracic Pads

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The thoracic pads can be adjusted independently in three directions, vertically, laterally and horizontally.

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To move the pads vertically and side to side turn knob “1” (in the picture above) anticlockwise then move the pad to the desired position. Lock the pad in place by turning knob “1” clockwise until it is tight.

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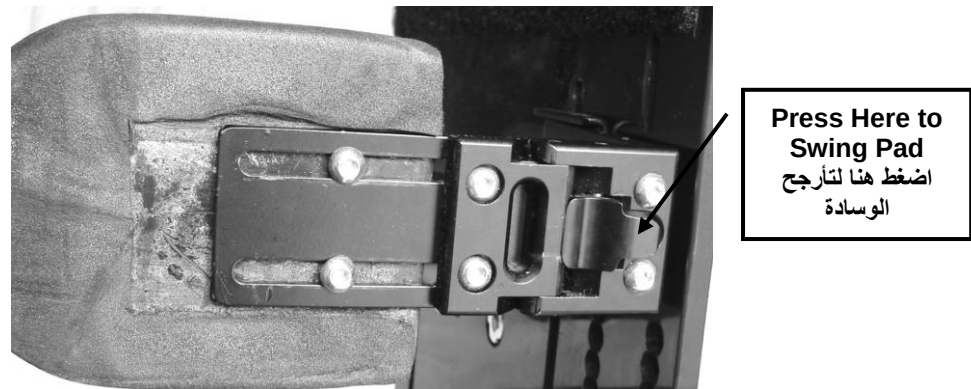
To move the pad forward and backwards, slide the cover forwards and loosen the two screws on the side of the pad “2” (in the picture above) using a 4 mm Allen Key. Move the pad to the desired position. Retighten the screws to lock the pad in place. Slide the cover back and press Hook & loop seal together.

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NOTE - Swing-A-Way thoracic pads are available as an optional extra to provide easy access to the seat. If these are fitted simply press the catch on the hinge to swing them aside. (See the picture below)

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II: Slot the mounting clips into the receiver brackets on the frame, and push through until the retaining clip locks into position (see Fig 7.2, right).

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Check the clearance above the headrest. If there is enough clearance skip to step V. If more height is needed, follow steps III and IV below to adjust the mounting height of the sunhood.

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III: If the height needs adjusting, unclip the sun hood and then use a 3mm hexagon key to loosen the 2 mounting screws as shown (see Fig 7.3, left).

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IV: Slide the mounting brackets up the frame tubes to the desired position, then fix in place by tightening the fixing screws using a 3mm hexagon key. Take care to ensure both brackets are adjusted to the same height.

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Fig 7.4 right: Shows the brackets adjusted to a mid-position on each side of the frame.

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V: Clip the sun hood to the frame, then loosen the thumb screws on the sun hood and open to the desired position. Once in the required position fix in place by tightening the thumb screws.

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VI: Position the leg cover over the occupant and secure to the sun hood by hooking four of the toggles (two each side) into the elastic loops in the sun hood as shown (see Fig 7.6). Please note, there are multiple toggles on the leg blanket to allow for different sizes of seat. Select the positions that provide the most coverage for the occupant.

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NOTE: Due to the wide range of shapes and sizes of modular seats and variety of bases, it may not always be possible to position the leg cover to give full protection on all systems.

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VII: Fig 7.7, left: Shows the leg cover attached to the sun canopy.

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VIII: Hook the elasticated pocket on the bottom of the leg cover under the foot rest as shown (See Fig 7.8, right).

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IX: Fig 7.9: Shows the sun canopy and leg cover fitted to a modular on a Snappi Size 1 chassis.

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Important Points When Using The Sunhood & Raincover

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8.2 Regular Inspections فحص دوري

Should a problem be found when carrying out the regular checks, it should be immediately reported to the issuing authority or Tendercare Ltd.

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The user's family can easily carry out the following tasks

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The item should be stored in a dry environment to avoid damage to any parts. Please ensure you do not store anything on top of the item as this too could cause damage

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When a product reaches the end of its lifetime, it should be returned to the authorised distributor for their review and disposal. For products which have been purchased privately, the product should be separated by material type so the parts can be recycled or disposed of properly. If required, please contact for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally friendly way.

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