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Snappi Pushchair

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1: Who to contact in difficulty

Please contact your local, authorised distributor if you have any questions regarding the use, maintenance or safety of your pushchair. In case there is no authorised distributor in your area or you have any questions, kindly contact:

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Tendercare Ltd.
Unit 10 Minster Court, Courtwick Lane, Littlehampton, BN17 7RN

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Email: info@tendercareltd.com

Web: www.tendercareltd.com

If there is a serious and unexpected accident with the device, please immediately inform both Tendercare Ltd and the responsible authorities.

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The quality management system of Tendercare Ltd is certified to ISO 9001:2015 and ISO 13485:2016.

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2: Introduction

The Snappi is a highly adjustable pushchair designed to accommodate a wide variety of seating needs. The Snappi pushchair is a Class 1 medical device designed for regular use. The device can be used by children with disabilities who lack the ability to maintain a seated position. The target group for the device is based on functional ability and not a specific health condition or age. The pushchair system comprises a wheelbase and a seating unit which is easily removable for storage. This pushchair is designed to be used as a mobility aid and not as a static seating system; as such, the occupant should not be left in the chair without an attendant holding the push handles. The product can be used at a walking pace both indoors and outdoors. The Snappi pushchair (forward facing pushchairs ONLY) can be used for transport in a wheelchair adapted vehicle in accordance with ISO 7176-19 (please see section 13 'Transporting the Wheelbase' for full instructions). The pushchair is available in 3 sizes: size 1 with minimiser, size 1 and size 2. A wide range of accessories are available for the Snappi pushchair to enable customisation to an individual's specific requirements.

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Risks Associated with Use and Contraindications

If used properly, risks can be eliminated during use. There are no known contraindications.

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Seat adjustments (please see table below for details of adjustment ranges): Seat depth adjustment, depth and length adjustable Hip Guides, Back Height adjustment, independent back recline (using an easy to operate gas strut mechanism) and depth & angle adjustable Footrest. The seat also includes an interface allowing it to be quickly fitted and removed from the wheelbase, and like the entire Snappi range, includes all the latest safety mechanisms (including an innovative 2 stage release to prevent little fingers causing accidents).

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A child's safety is your responsibility. As such we recommend that you read the complete user manual prior to using your pushchair.

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IMPORTANT:

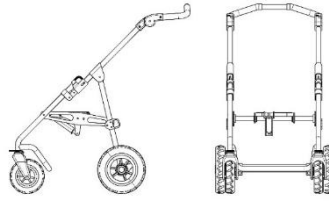
* Knee angle adjustable in 15° increments

** Maximum carry weight is defined as the user weight, plus any accessories / equipment which may be fitted or carried on the chassis. The weight of the Snappi seat has been taken into account with these weights and you do not need to make any deduction for the Snappi Seat. Warning: Never exceed the maximum stated carry weight.

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Please note, the exact length of depend upon what back height

your pushchair when folded will your seat is set to. We have taken the measurements both when the seat is set to its minimum and maximum back height. The size of your folded Snappi pushchair will be within this range.

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All sizes and weights are given as a guide. Tendercare Ltd reserves the right to amend specifications at any time as part of their product development programme.

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3: Unpacking

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The wheelbases and seat unit are delivered together in a cardboard carton.

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WARNING:

The transit carton is quite bulky so moving and unpacking must be done with care. Observe all lifting and handling regulations.

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Stand the carton upright making sure it is supported and cannot fall over. Open the carton and remove any packages or packing, which could obstruct the removal of the seat and

wheelbase. Remove the seat first, then the wheelbase. Do not attempt to lift both parts out together.

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The carton should contain the following items:

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Please note accessories, such as lateral supports or harnessing that were ordered at the same time as the pushchair, will be included in either the main package or a separate carton.

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IMPORTANT:

- ❖ If any items are damaged or missing, then please contact Tendercare, preferably by email at info@tendercareltd.com or alternatively please call us on (01903) 726161 within 36 hours of delivery.

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After unpacking and checking you have all components and they are in good condition dispose of the packaging at your local recycling centre. Alternatively retain and reuse.

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3.1 Labels / Model Identification

The below table shows the labels found on your product. Depending on your issuing country, you will have either UKCA label/labels or CE label/labels.

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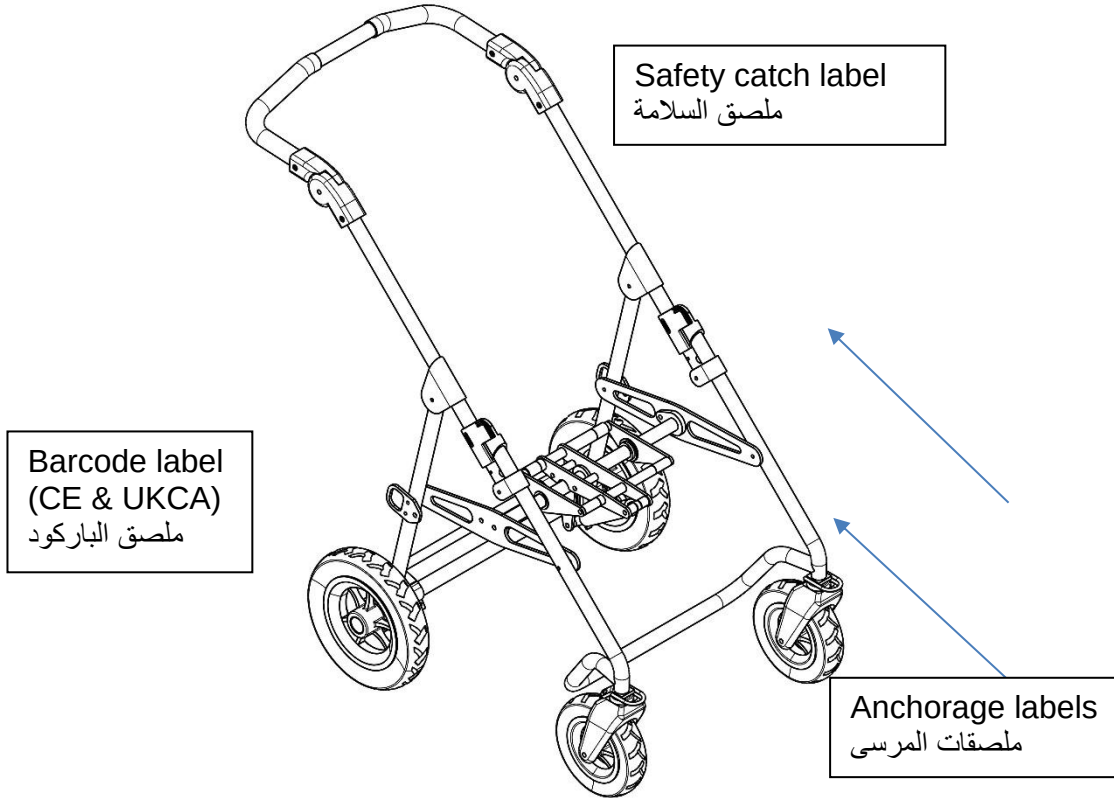
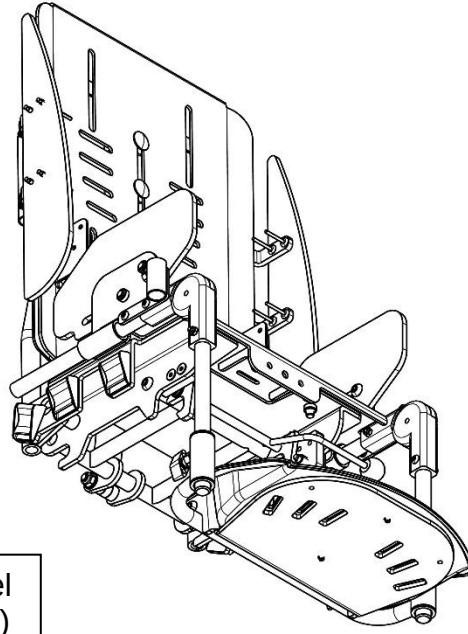
WARNING:

When opening or folding the wheelbase, ensure that you hold the frame so that you avoid any danger of catching your fingers in moving parts.

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Keep children clear of the wheelbase during opening and folding.

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To fold the pushchair, you first need to remove the seat.

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4.1.1 Unfolding the Frame (original locks)

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Standing at back of the wheelbase, place your foot on the rear cross member X (see Fig 4.1.1.1) and lift handle labelled Y as far as it will move.

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Next locate the 2 plastic locking sliders (See Fig 4.1.1.2). These will be stowed in the "open" position and are held in place by 2 small spring clips.

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Fig 4.1.1.3

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Press the 2 silver "Spring pins" in and slide the 2 locking pieces down over the lower tube sections. (See Fig 4.1.1.3)

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
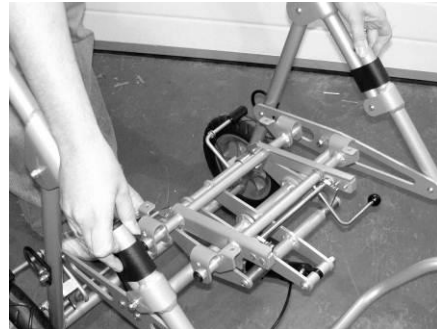
Fig 4.1.1.4

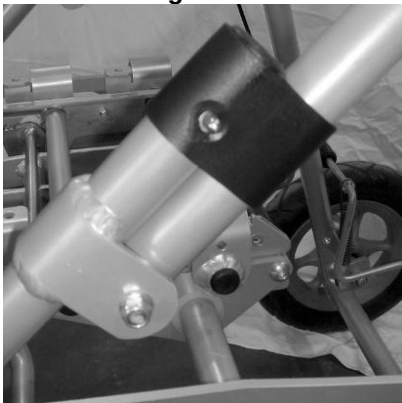The pins will spring through the holes in the sliders and hold them in place (see Fig 4.1.1.4).

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WARNING:

If the locking sliders are not in the correct position, or the spring pins do not protrude thereby allowing the locking clips to move back up the frame, then the frame may collapse in use.

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Folding the wheelbase: To unlock the frame and fold the wheelbase, reverse the above instructions.

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Standing at back of the wheelbase, place your foot on the rear cross member X (see Fig 4.1.2.1) and lift handle labelled Y as far as it will move.

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This will open the frame.

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Once the frame is almost open, the 2 plastic frame locks will engage with the angled sections of the lower frame.

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Continue to unfold the frame; this will force the locks up the tubes. There will be some resistance, as you will be working against the locking springs.

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
Once fully open, the frame locks will snap shut over the front frame, securing the frame in the open position.

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Always check that both locks are fully closed. In the locked position the guide bolt 'B' will be sat in the top of the slot 'A' on the side of the slider as shown (see Fig 4.1.2.3). If there is any slot visible above the guide bolt, push the lock down by hand until it will not move any further.

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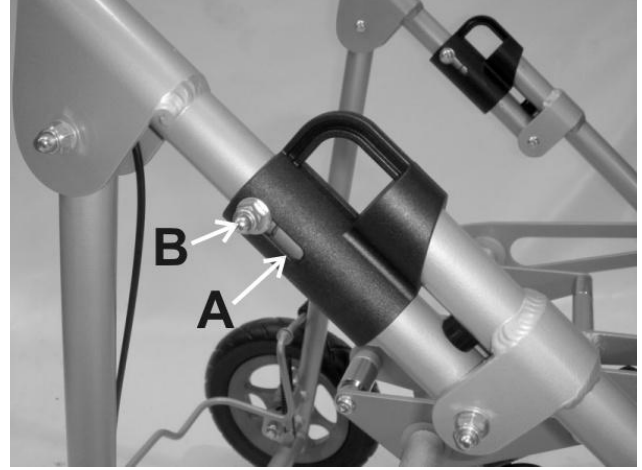Standing beside the frame, pull up the 2 locking sliders as shown (see Fig 4.1.2.4).

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Whilst holding the locks open, push down with your arm on the upper frame or push handle, so that the frame starts to fold.

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Release the locking sliders, and fold down the frame by moving the push handle down as far as it will move.

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IMPORTANT:


Always check that BOTH frame locks are fully closed before using the frame. If they are not properly engaged, the frame could collapse during use.

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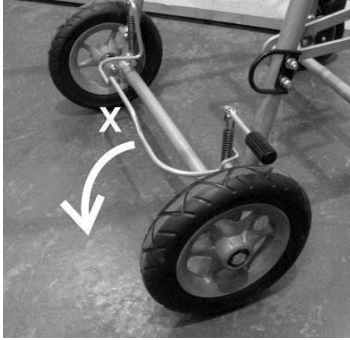To apply the brake, put your foot on the top of brake bar (labelled X) and push down as shown. The brake will flip down onto the wheels and lock them.

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To release the brake, hook your foot between the middle raised portion of the brake bar, and the frame cross bar. Then, lift the brake bar with your foot until it flips back and stops against the brake stop pins. We recommend always using solid footwear when operating the brake. Please note you may find it easier to use your hand to release the brake. If operating the brake by hand always hold the brake by the raised section in the middle of the bar.

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Important:

Always put the brake on when placing the child in, or taking them out of the pushchair. Do not leave the brake on when the pushchair is not in use as this will damage the rear wheels.

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WARNING:

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The brake mechanism is spring loaded so care must be taken when operating it.

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When operating the brake mechanism ensure that you always keep a firm hold of the pushchair, keeping your hands well away from the brake mechanism.

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Do not leave the pushchair on a slope, even with the brake on. Always ensure that the pushchair is on level ground to prevent risk of injury to the occupant and help prevent tipping.

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Do not operate the brake when the pushchair is moving. The brake is not designed to slow the pushchair down or stop a moving pushchair; such use may damage the brake.

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4.2B Pin Brake (pushchairs supplied after June 2015)

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Snappi Pushchairs supplied after June 2015 will be fitted with the new Pin Brake mechanism.

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To set the pin brake, first securely hold the push handle. Place your foot on the top of the push bar as shown (see Fig 4.2.3, left) then press the bar down as far as it will go. If the brake pin will not engage, try rolling the pushchair forward slightly to help line up the pins with the locking rings on the wheels.

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Fig 4.2.4 Right: The brake in the fully locked position.

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Fig 4.2.4

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To release the pin brake, hold the pushchair and place your foot underneath the brake bar as shown (see Fig 4.2.5, left). Lift the bar as far as it will move to ensure the pins are clear of the locking rings.

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Warning:

When operating the brake mechanism ensure that you always keep a firm hold of the pushchair and keep your hands well away from the brake mechanism.

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4.2C Tyres:

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As standard the Snappi pushchair is supplied with solid tyres, however pneumatic tyres are also available as an option. If your pushchair is supplied with pneumatic tyres these need to be regularly checked and inflated if necessary using a hand or foot pump. The operating pressure for the tyres is 36 PSI (note: do not inflate the tyres above 36 PSI). Do not use high pressure air lines or mechanical pumps as used in garages, to inflate the tyres.

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4.2D Push Handle Mounted Attendant Brakes – Hub (optional upgrade):

Hub brakes are available only as a factory fitted option. The following instructions detail how to operate the hub brakes.

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NOTE: The hub brake upgrade means the standard foot operated brake is removed and is not available on pushchairs fitted with hub brakes. The hub brakes include a locking function that provides the same functionality as the standard parking pin lock brake.

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Above: The Snappi Chassis fitted with hub brakes.

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Note that there is no longer a brake bar between the rear wheels. The hub brakes are operated by the pair of locking levers positioned on the push handle.

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To set the brakes, squeeze the brake levers on both sides as shown (see Fig 4.2.7, right).

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Fig 4.2.7

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The brake leavers feature an automatic lock mechanism and will lock with the brakes on.

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The system features 3 locking positions with different levels of braking force. We recommend to always use the 3rd (strongest) position to ensure occupant safety.

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Fig 4.2.8

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To release the brake lift the smaller release catches on the handles to release the automatic lock as shown (see Fig 4.2.9, right).

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Fig 4.2.9

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Note:

Do not leave the pushchair on a slope, even with the brake on. Always ensure that the pushchair is on level ground to prevent risk of injury to the occupant and help prevent tipping .

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Do not attempt to push the pushchair when the brake is locked off. Doing so may damage the brake mechanism. If you need to slow down the momentum of your pushchair (i.e. when on a hill), you must not lock off the brake; rather, you must keep your finger pressed on the automatic lock as shown in Fig 4.2. 9, above, to ensure the lock does not engage.

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4.2E Front Locking Castors (Optional)

Front locking castors are available as option upgrade to the wheelbase. These are a factory fitted option, so should be included on the order at time of purchase. Alternatively, the pushchair can be returned to the Tendercare factory for fitting if ordering for an existing pushchair.

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Then rotate the front castor until the pin engages with the hole. The castor should now be locked in a forward position (see Fig 4.2E.2). **Note: front locking castors are generally used on uneven/rough terrain so for day-to-day use, when the castors are locked, it may be more difficult to change direction.**

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Castor Locking Pin (in a locked position)

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The seat incorporates an Interface system, and includes a 2-stage latch that comprises a main latch and a second safety latch that prevents accidental release of the seat from the chassis.

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The safety catch must be correctly set before the main catch can be operated:

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
Fig 4.3.1

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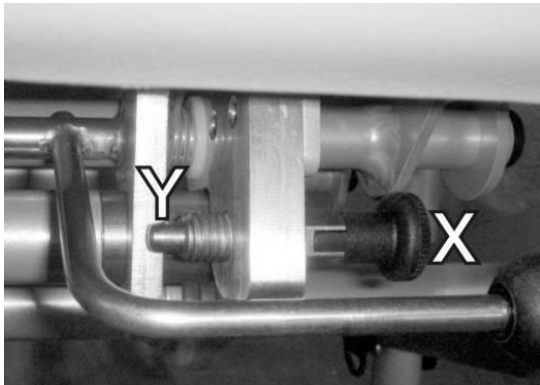The safety catch clip is located under the front right corner of the seat near to the release lever.

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
To release it, pull the handle "X", and turn it 90 degrees so that the pin "Y" is fully retracted.

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To lock it, pull "X" and turn 90 degrees in the opposite direction, so that pin "Y" is fully protruded and sits behind the interface main catch "B" (see fig 4.3.3 on next page).

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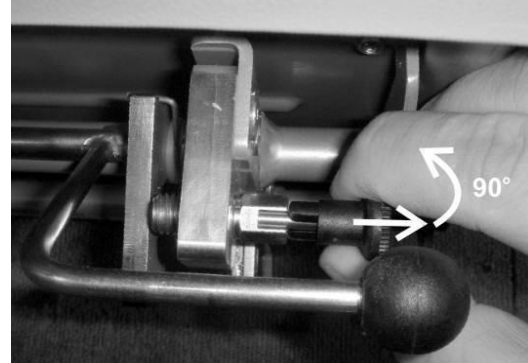Important:

When fitting or removing the seat, the carer / parent must first ensure that the safety catch pin is set in the "unlocked" position. It will not be possible to fit or remove the seat if the latch is "locked", and attempting to do so may cause damage to the chassis or the seat.

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Once the seat has been fitted, the carer / parent must always lock the safety catch.

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Note:

A warning label is positioned on the front right hand portion of the tilt frame on the chassis, to remind carers to lock the safety catch.

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Definition of interface mounting points:

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Ensure that the safety catch "C" has been unlocked (see instructions on previous page).

Holding the seat as shown, tilt it back and locate the 2 rear hooks "A" on the back of the interface, over the stainless steel bushings on the back of the frame interface "E".

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Next, rotate the seat forward and down so that the main clips "B" snap over the front stainless steel bushes on the frame interface "F".

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Note that due to the design of the frame interface, it is not possible to fit the seat backwards. If fitted the wrong way around the clips will not close and it will not sit down into the frame.

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Next, push down firmly on the release lever "D" to ensure that the clips are fully closed. Note the lever is positioned under the front right corner of the seat (see Fig 4.3.7).

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Set the safety catch "C" to the "Locked" position (so that the pin sits behind the interface clip).

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To remove the seat, first unlock the safety catch "C" as detailed above.

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Lift the release lever "D" up, whilst holding the top of the seat with your other hand. This will open the main clips "B". Rotate the seat back so that the main clips are clear from the frame, and lift the seat away from the buggy.

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IMPORTANT:

- ❖ *Always ensure that the seat interface clips are fully closed. To test this, press down on the release lever when fitted and make sure the clips will not move (see fig 4.3.5).*
- ❖ *Always lock the safety catch before using the buggy.*
- ❖ *Always unlock the safety catch before fitting or removing the seat. Attempting to fit or remove the seat with the catch locked could cause damage to the seat or the chassis.*
- ❖ *Do not attempt to fit the seat backwards. It will not lock in a rearward facing position.*
- ❖ *The seat interface system is spring loaded, so ensure your fingers are clear of the clips when fitting or removing.*

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
Fig 4.4.1

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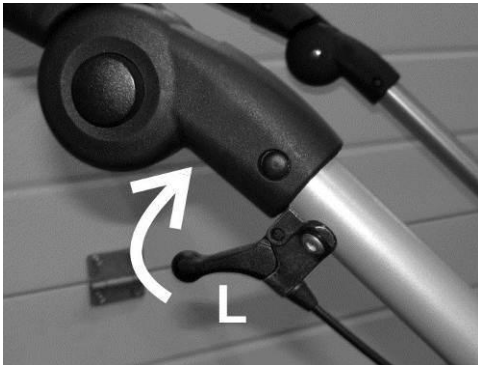To operate the tilt in space mechanism, place your hand over the push handle adjustor, and lift the lever 'L' (see Fig 4.4.1) as shown.

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Fig 4.4.2

This will open the gas strut and tilt the seat.
Note; The speed of tilt can be controlled by varying the pressure applied at lever L.

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Use your other hand to support seat as it tilts.

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To reset the tilt mechanism, hold down lever L and push down firmly on the front of the seat as shown (see Fig 4.4.3). Once in the desired position, release lever L. For a 90-degree position, move the seat so the gas strut is fully closed and it will not move any further before releasing the tilt lever.

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IMPORTANT:

- ❖ Always ensure any harness provided for the child is used and correctly adjusted before reclining or returning the seat to a more upright position.
- ❖ Always support the seat when tilting, as the gas springs can be quick to operate. If the seat is not supported, it may move swiftly and could cause the occupant distress.

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First, set the seat to full tilt in space as detailed in section 4.4

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Next fold the chassis as described in section 4.1 without removing the seat.

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Next, set the seat back angle to its maximum position as shown (see instruction on section 5.1.3).

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Finally, fold the footrest, and tuck it in using the knee angle adjusters as shown (see section 5.1.2 for details).

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4.6 Folding as a 2 part system (all models) – Including rearward facing model

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To fold the system as a 2 part, first set the seat back angle to 90° (see section 5.1.3), remove the seat and fold the frame as detailed in section 4.1.

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Next place the seat, face down over the folded chassis as shown.

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Finally, fold the footrest tray and angle the footrest tubes to sit closer to the seat (see section 5.1.2 for how to adjust and fold the footrest).

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When setting up the seat unit always ensure the following is done:

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Note: you can now adjust the seat to suit your child's dimensions

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Common adjustments on the Snappi Seat can be made by hand (such as back height and back recline angle) and require no tools. Less frequent adjustments are made using a 5mm hexagon key (supplied), such as the seat depth and width. No other tools are required to make adjustments to the seat.

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5.1.1 Seat Depth and Hip Guides

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To adjust the seat depth and width, first remove the seat base cover (this is attached by Hook & loop in the centre of the seat and is easily removed by pulling it away from the frame).

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Using the supplied 5mm hexagon key, loosen the 4 seat fixings.

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Next set the seat width ('A' see Fig 5.1.1.1) as measured in section 5.0 by sliding the hip guide brackets in or out as required.

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To adjust the seat depth ('B' see Fig 5.1.1.1), slide the upper of the base plates forward or back to achieve the required position.

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Once adjusted, tighten the 4 bolts using the hexagon key and fit the cover back into place.

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The Snappi hip guide also offers depth adjustment ('C' see Fig 5.1.1.2) to best suit your child's needs.

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Once adjusted, tighten the 2 fixings and refit the hip guide cover.

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5.1.2 Footrest Depth and Knee Angle Adjustment

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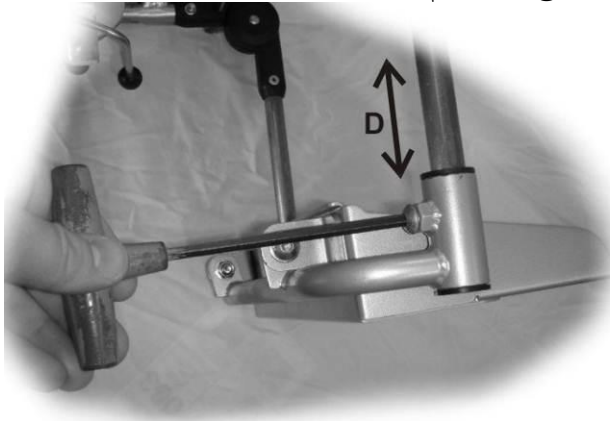
Fig 5.1.2.1

To adjust the foot depth, loosen the 2 securing bolts (size 1) using the 5mm hexagon key as shown (see Fig 5.1.2.1). On some size 2 models, wing nuts may be fitted in place of bolts, and therefore the adjustment can be made by hand.

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Slide the footrest up or down the guide tubes to the required height as measured in section 5.0

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Once set, lock the position by tightening the 2 fixing bolts / wing nuts.

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To adjust the knee angle, press in the angle adjusters on both sides of the footrest at the same time (see Fig 5.1.2.2), and lift the footrest guide tubes to the required angle. To lock, release the buttons.

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Note that the adjusters can only be locked in 15° increments.

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For the rearward facing Snappi it may be necessary to reduce the length of the footrest stems to avoid the footrest catching and inadvertently operating the brake bar. Alternatively the buggy can be placed in some degree of tilt and space to avoid this situation.

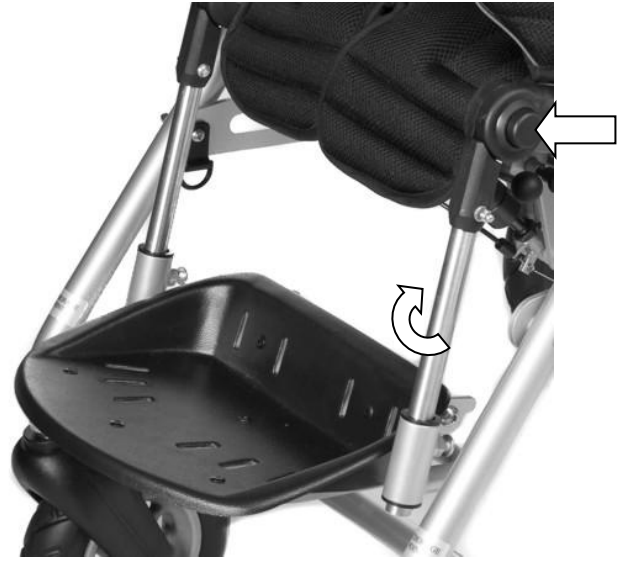
Fig 5.1.2.2

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To adjust the back height, first un-tuck the cover from the bottom of the seat (the cover doesn't need to be removed).

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Next, turn the 2 large central hand wheels on the back plate, as shown in Fig 5.1.3.1, anticlockwise by either half a turn or, if necessary, one full turn.

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Once these are loose, raise or lower the upper section of the back to the required height, and turn the 2 hand wheels clockwise until tight to lock it back into place.

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Once adjusted, tuck the cover back around the bottom of the seat with the excess sitting between the lower back plate and the gas spring.

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N.B. Before adjusting the back height you may need to remove any lateral trunk supports and/or the top straps of any chest harnessing.

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To adjust the back angle, grip the adjustment lever and lift it up, whilst holding the rear cross bar with your thumb. (See Fig 5.1.3.1)

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To recline the seat, pull the top of the seat backwards to the required angle, and release the adjustment lever. For seats fitted with a dynamic gas strut, the dynamic strut restrictor will reduce the amount of seat recline (see Fig. 5.1.3.2). If full recline is needed, the restrictor needs to be removed (see Snappi Pushchair Workshop Manual about how to do this).

Fig 5.1.3.2

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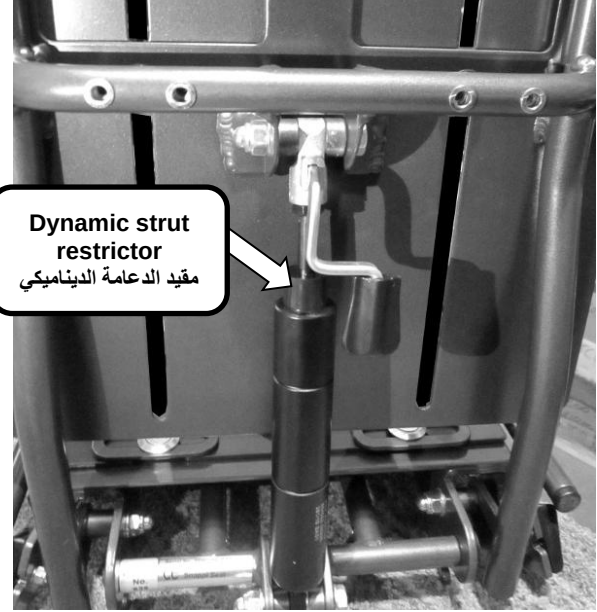
Warning: the restrictor must be in place if this Snappi is to be transported.

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To bring the seat back to a 90° position, pull the release lever and gently lift the seat (the gas spring will assist in returning the seat). Once at the required angle, release the adjustment lever.

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The lateral trunk supports are supplied with two M8 coach bolts, and two 30mm hand wheels per bracket (see Figs 5.1.4.1 and 5.1.4.2).

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The laterals are fitted by securing with coach bolts and hand wheels, through the mounting slots 'H' (see Fig 5.1.4.3) on the back of the seat unit.

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The lateral bracket must be fed through the slot in the cover (see Fig 5.1.4.4).

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Next, insert the 2 coach bolts through the bracket and seat back plate, so that the square neck is sitting fully in the slots.

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Finally, secure the laterals by screwing the provided hand wheels onto the protruding threads of the coach bolts (see Fig 5.1.4.5).

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The position of the laterals can be adjusted laterally by loosening the hand wheels and sliding them in or out to the required width.

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They can be adjusted vertically by sliding up and down the mounting slots 'H'(see fig 5.1.4.3). There is a partition in the slots between the upper and lower back plates. To achieve the full adjustment range, the laterals need to be fitted in the appropriate section of the slots.

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Simply lift the front edge of the seat pad to separate it from the Hook & loop strip on the base.

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The cover hooks around the front of the hip guide plate. To remove, lift and rotate the cover from the rear as shown, and then unhook the cover from under the front of the hip guide plate.

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Undo the 2 'popper' fasteners, (See Fig 5.2.3) and unpeel the Hook & loop securing the pad.

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Undo the 2 'popper' fasteners located on the top left and right corners of the seat (see right).

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Next unhook the top of the back pad from the seat, and then un-tuck the material from around the bottom of the seat back plates to release the cover.

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5.3 Fitting a Headrest Pad or Lateral Head Supports (optional)

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Fig 5.3.1

There are three types of one piece headrest available; standard, extra recess (Fig 5.3.1) and occipital roll, however these can all be fitted to the seat in the same way.

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The headrest pads include a pair of Hook & loop securing straps.

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Fig 5.3.2

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The headrest is fitted to the cover, through the upper fitting slots (see Fig 5.3.2).

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Fig 5.3.3

First, remove the seat back pad from the frame (see section 5.2).

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5.2

Next, feed the Hook & loop straps though the slots, and join together to secure the headrest in place.

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Finally, refit the seat back pad to the seat frame.

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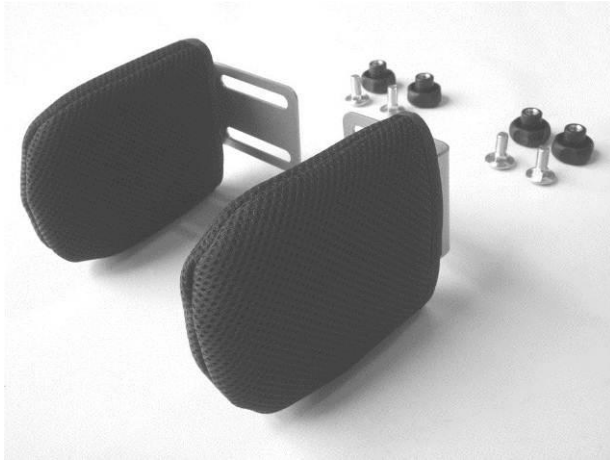
Fig 5.3.5

The lateral head supports are supplied with two M8 coach bolts, and two 30mm hand wheels per bracket (see Fig 5.3.5)

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The lateral head supports are fitted by securing with coach bolts and hand wheels, through the mounting slots 'I' (see Fig 5.3.6) on the back of the seat unit.

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The position of the laterals can be adjusted laterally by loosening the hand wheels and sliding them in or out to the required width. They can be adjusted vertically by sliding up and down the mounting slots 'I' (see fig 5.3.6).

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Fig 5.3.6

5.4 Fitting the Pelvic Strap (recommended on all chairs)

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A 2 point pelvic belt should be fitted to the seat such that the 2 padded areas are facing the back of the seat (so that they will be against the child when he / she is in the seat)

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To fit the 2-point pelvic strap to the old-style Snappi, (which can be distinguished by having only one cam-lock buckle on each side):

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Open the cam-lock buckles (these are located on the bottom left and right corners of the seat base).

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Feed the webbing of the strap through the cam lock. (See Fig 5.4.3)

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Adjust the length of the pelvic strap by pulling the webbing through the cam buckle until it is at the required length. (Fig 5.4.4)

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Fig 5.4.4

Once adjusted, lock the cam buckle on both sides to secure the strap.

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Important: The pelvic strap must be located as low down over the hips as possible. It must not cross the stomach of the passenger.

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To fit the 2-point pelvic strap to the new-style Snappi, (which can be distinguished by having 3 cam buckles on each side):

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Open the middle cam-lock buckles (these are located on the bottom left and right corners of the seat base).

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Feed the webbing of the strap between the seat plate and seat depth adjustment tube, and through the cam lock. (See Fig 5.4.5)

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Adjust the length of the pelvic strap by pulling the webbing through the cam buckle until it is at the required length. (Fig 5.4.6)

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Once adjusted, lock the buckles on both sides of the seat to secure the strap.

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To fit the 4-point pelvic strap to the new-style Snappi, (which has 3 cam buckles on each side):

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Open the middle and front cam-lock buckles.

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Feed the top webbing straps (See Fig 5.4.2) between the seat plate and seat depth adjustment tubes and through the middle cam lock. (See Fig 5.4.5)

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Feed the bottom webbing straps through the slot at the bottom of the Hip Guide plates, over the seat depth adjustment tubes and through the front cam lock. (See Fig 5.4.7)

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Fig 5.5.1

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The Groin Adaptation strap is a piece of webbing which can fit to the pelvic strap or the H belt harness to help with the positioning of the child. It is designed to help prevent the child from slipping forward underneath the harness.

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Feed the loop end of the strap up through either of the two slots in the base of the seat. (See Fig 5.5.3) The slot that should be used will be determined by the size of the child. For larger children the slot closest to the front of the seat will probably be more suitable, whilst for smaller children the second slot in would most likely be appropriate. Please note you may need to move the 3 bar slider down the strap.

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The pelvic strap, or lap belt of the H-belt harness should then be threaded through the loop end of the groin adaptation strap. (See Fig 5.5.2) Once the strap is in place, and you have comfortably positioned your child, slide the 3 bar slider up so that it is as close under the seat as possible. This will keep the strap in place.

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The Butterfly harness comprises a padded section that sits against the user's chest, two shoulder and two bottom straps. Either the shoulder or the bottom straps are locked in a position to suit the user and then final adjustment is made on the opposite straps.

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The harness is designed to provide comfortable postural support for the user.

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Fig 5.6.1

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Alternatively an H-Belt harness is available, which comprises 4 padded straps that join together with a quick release buckle to provide easy open and release of the harness.

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Important: The harnesses fitted are for postural support only and must not be used as a vehicle harness.

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Fig 5.6.2

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5.7 Final checks

1. **IMPORTANT:** In order to ensure that the Butterfly (Chest) or H-Belt harness cannot rise up and cause harm to the user, every time the harness is in use, you must ensure that all cam buckles/fixings are properly closed and that the harness is correctly positioned a minimum of 75mm below the user's neckline. Failure to comply with this could result in serious injury.

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4. Check that the seat unit is located and locked correctly in the chassis.

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6. Position the child correctly in the seat, and assess the child's posture, if the child's posture appears to have changed remove the child and compare the support positions to the measurements taken in section 5.1. Adjust the supports as required, and reseat the child.

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7. Check footplate position and adjust if necessary.

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6: Accessories

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6.1: Shopping Basket

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Fitting / removing the shopping basket:

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There are 4 hanging points located on the insides of the side plates (see left).

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To fit the shopping basket, clip the snap hooks on the basket onto the 4 clipping points.

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To remove the shopping basket, simply unclip the 4 hooks.

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The shopping basket includes an expandable section at the back to help with loading of bulky items. To expand the basket, simply peel apart the Hook & loop at the back to release the extra material.

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The equipment-carrying tray has 2 sets of fixing points on its underside to secure it to the chassis (see right).
تحتوي صينية حمل المعدات على مجموعتين من نقاط

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Feed the tray under the wheelbase from the front of the chassis and hook the rear points 'A' under the rear cross bar, ensuring that the tray is in the middle so it sits in-between the vertical brake bars.

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Next, with the front turn catch 'B' rotated out of the way; rotate the tray down so it sits on the front cross bar.

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Finally turn the front catch 'B' 90 degrees so it sits under the front cross bar (this catch may be quite tight).

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To remove the tray, unlock catch B, lift up and remove the tray from the chassis.

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To fit the Umbrella Style sun canopy:

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Remove the umbrella from its protective sleeve.

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Next, unclip the frame fixing from the umbrella by pressing on the button and pulling apart as shown in Fig 6.3.2

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Next, unscrew the frame mounting to open the jaws until it will fit over the frame.

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Position the mounting onto the upper frame tube (see fig 5.3.4 below).

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Adjust the angle of the clip so that it is vertical.

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When in the desired position, retighten the screw to lock it onto the frame

Finally, clip the umbrella section back onto the mounting by pushing the 2 halves of the clip back together, and open the umbrella.

If a different angle is needed, simply loosen the locking screw, move to the required angle and retighten.

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To fit the plastic rainshield, unfold it from its packaging and place over the sunhood with the hook & loop rainshield flap facing the front (i.e. world as opposed to occupant facing). The sunhood angle needs to be set to match the seemlines to those on the rainshield. Next tuck the elasticated edge under the footrest as shown in Fig 9, right. **Note, you may not be able to use back recline at the same time as the rainshield (tilt in space can still be used).**

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The rainshield has a back flap to minimise wind and rain infiltration. To keep the back flap in place, tuck this down inside the push handle and fasten the hook & loop fixings which are located on both the back flap and the main skirt of the rainshield. It should now look like Fig. 10, left,

Once you have stuck the hook & loop together, you may need to adjust the rainshield to help ensure that the rain shield is straight (See Fig 10).

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Important Points When Using The Sunhood & Rainshield

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- In warm conditions (e.g. inside building, warm days etc.), ensure you undo the hook & loop flap at the front of the rain shield and fold it back to prevent the occupant of the pushchair from overheating. On some occasions undoing the flap may not be sufficient to keep the occupant cool, as such you may need to completely remove the rainshield.

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- We do not recommend ever leaving the child unattended when using the pushchair. Additional care should be taken when using the sunhood and transparent rainshield, to ensure neither the sunhood nor rainshield are too close to the child's face.

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6.5: Rain Cover – not suitable for rearward facing Snappi

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This is the recommended rain cover if you need to use back recline and a rain cover at the same time.

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Attach the 4 push-on clips to the upper frame of the pushchair, with the open clip section pointing downwards.

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The lower clip fits directly above the nylon locking slider, while the upper clip attaches to the upper frame tube.

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Feed the open flap of material around the rear frame, inside the push handles.

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Next fold the cover down, and secure the bottom using the 2 elastic 'C' clips.

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Finally, join the rear flap of material to the sides of the rain cover with the Hook & loop tabs.

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To fold the cover back when not needed, simply unhook the 2 elastic 'C' clips and fold back. The cover can be secured in the open position by attaching the C clips to the upper frame.

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7.2 Getting out of the Snappi Pushchair

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8.1 Pushing

Important:

- ❖ *At the start of every journey make sure that the interface is correctly fitted into the frame and that the locking clips are fully closed. Please refer to section 4.3 'Fitting the seat to the chassis' for more information.*

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❖ *Do not allow children to play with either of the gas spring release levers.*

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1. The wheelbase is easy to push over hard, even surfaces but may be more difficult over rough, uneven surfaces and the wheels can sink into soft ground. **Never force the wheelbase if it becomes stuck**, this may cause the rear wheels to lift and the wheelbase to be tipped forwards. If the wheelbase gets jammed on a stone or cracked pavement, tip it backwards to raise the front wheel over the obstacle and then proceed. On soft ground it may be easier to pull the wheelbase backwards.

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2. The wheelbase has swivelling front castor wheels and is easily steered simply by turning the wheelbase in the desired direction.

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3. Whenever possible avoid steep and uneven surfaces because there is a danger that the wheelbase may tip over. If you cannot avoid difficult surfaces you must take extra care when travelling across them.

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4. If your pushchair is fitted with the optional pneumatic tyres, always check that no sharp objects are in your path to avoid the risk of a puncture.

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5. Do not use the wheelbase in sandy conditions.

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6. Always use two hands for pushing the wheelbase.

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7. For the comfort of the occupant always avoid pushing too fast or making sudden fast turns, jerking, jolting and bumping. Many occupants are liable to be affected by rough handling, which can cause loss of confidence, discomfort or pain.

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8. Avoid kerbs. Wherever possible use ramped kerbs or slopes. This will make pushing the wheelbase easier and safer and will be more comfortable for the occupant.

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9. Avoid steep kerbs and flights of steps because of the danger of losing your grip on the wheelbase or of the occupant being tipped out. Always use a ramp or lift if available. Do not push the chair down flights of stairs as this puts strain on the pushchair and puts the occupant at risk.

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10. To tip the wheelbase backwards when climbing kerbs, stand behind and whilst holding onto the push handle with both hands, press down and pull backwards. This will lift the front of the wheelbase.

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11. Always look well ahead so that you have plenty of time to avoid hazards.

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12. Be considerate to other people and watch out for pedestrians who may stop or suddenly change direction.

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13. It is the responsibility of the person pushing the pushchair to be always aware of their surroundings and to be aware of any potential hazards and dangers

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8.2 Brakes

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Always put the brake on when placing the child in or taking them out of the seat.

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Do not leave the pushchair on a slope, even with the brake on. Always ensure that the pushchair is on level ground to prevent risk of injury to the occupant.

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Do not leave the brake on when the pushchair is not in use as this will damage the rear wheels.

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10: Maintenance

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Should a problem be found when carrying out the regular checks, it should be immediately reported to either the authorised distributor (usually your wheelchair services) or to Tendercare Ltd.

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Below are a series of simple checks which should be regularly carried out as part of routine maintenance of the pushchair. Frequency of maintenance should be increased for heavier users. If you discover a fault or any signs of damage, immediately stop using the pushchair and contact either Tendercare Limited or your authorised distributor for further instruction. Do not attempt to correct any issues yourselves without further instruction from either Tendercare Ltd or your authorised distributor. If you discover damage or a fault which is not listed below, stop using the pushchair and immediately contact either your authorised

distributor or Tendercare Ltd. Similarly, if you have any queries or concerns, please contact either your authorised distributor or Tendercare Ltd.

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If a portable oxygen cylinder is required, the maximum size recommended by Tendercare is type PD 455 mm long x 100 mm diameter weight approximately 10 kg.

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12: Warranty, Product Life, Service Life & Reuse

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The Snappi pushchair has a period of use of up to 7 years if used correctly and serviced regularly by a professional as per the information detailed in this user manual.

The Snappi pushchair may be used beyond this point if it is in safe and technically sound condition. Time spent in storage at the manufacturer does not count towards the period of use. The period of use/ product lifetime clock commences from the date of despatch from our premises.

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The theoretical period of use does not equate to a guaranteed service life. It depends on the intensity of use of the product. The product must always have undergone an individual assessment by the authorised distributor (or their designated agent/repairer), each time the product is reused. The period of use does not apply to wear parts like padding, covers, tyres, brake pads, etc.

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12.3 Service Life

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A product's Service Lifetime is the amount of time that the manufacturer determines the product to be "serviceable" and for which it can function safely and effectively. The product's service lifetime clock commences from the date of despatch from our premises.

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The Snappi pushchair has a service life of 8 years if the requirements stipulated for the period of use are met. The service life consists of the period of use and the storage times.

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The Snappi pushchair is suitable to be reused. Before being passed on to the next user, the product always has to be carefully cleaned and disinfected. The condition has to be checked by trained staff for any wear and damage. The Snappi pushchair must always have undergone an individual assessment by the authorised distributor (or their designated agent/repairer), each time it is reused. Any issues have to be remedied to ensure the continued safe use of the product. A copy of the Instructions for Use (user manual) is a component of the product and has to be passed on to the new user.

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13: Repairs

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1. Repairs: For all repairs contact your authorised distributor.

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The item should be stored in a dry environment to avoid damage to any parts. Please ensure you do not store anything on top of the item as this too could cause damage.

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When a product reaches the end of its lifetime, it should be returned to the authorised distributor for their review and disposal. For products which have been purchased privately, the product should be separated by material type so the parts can be recycled or disposed of properly. If required, please contact for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally friendly way.

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