AVOVA Sperling Reha • AXKID Nextkid Recaro Monza CFX Reha


Adapterplatte (High-Tech Unterbau)

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Adapter plate (high-tech base)

Assembly instructions – Accessory

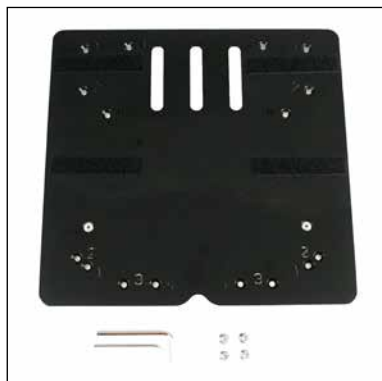1

2

3 - Montagepunkte (2)
(Axxid NEXTKID Reha)

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The adapter (Fig. 1) plate offers the option of mounting a wide range of accessories:

- **Adapter plate for footrest**
- **Adapter plate for footrest with seatfix**
- **Tilting base**
- **Swivel base**
- **Swivel and tilting base**
- **Seat depth extension (not for all rehab child car seats)**
- **Abduction block, adjustable (not for all rehab child car seats)**

The possible combinations with the adapter plate depend on your rehab child car seat.

You can find out what options your rehab child car seat offers in the instructions for use for your rehab child car seat or under this link: www.thomashilfen.com/product-instructions

Tip: Two Allen keys are supplied with the adapter plate (Fig. 2). The allen key size 3 is required for fitting the adapter plate, for example. The allen key size 4 is required, for example, if you are using other accessories such as the footrest.

Fitting the adapter plate to the rehab car seat

Place the car seat on its back. Position the adapter plate in front of the threaded sleeves on the underside of your rehab child car seat as shown in figure 3-5. The fleece strips do not lie against the seat! (Fig. 2)

Screw the adapter plate on tightly using the countersunk screws.

Installation in your vehicle

Tip: Before fitting in your vehicle, please push the front vehicle seat far forwards. This will give you more space in the vehicle for installation.

Install your rehab child car seat with the adapter plate in your vehicle as described in the user manual of your rehab child car seat. Push the rehab car seat as close as possible to the backrest of your vehicle. The rehab child car seat must have as much contact as possible with the backrest of your vehicle.

Removal from your vehicle

Please also follow the instructions in the user manual for your rehab car seat exactly when removing it from your vehicle.

Tip: Before you remove your rehab car seat from the vehicle, the ISOFIX connectors of your rehab child car seat should be pushed back under the seat to prevent damage to the vehicle.

Dismantling adapter plate

Once you have removed your rehab child car seat from your vehicle, lay it on its back to loosen the four screws on the adapter plate.

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG · Walkmühlenstr. 1 · D - 27432 Bremervörde · Germany