AVOVA Sperling Reha • Axxid NEXTKID Reha Recaro Monza CFX Reha

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This footrest adapter (Fig. 1) is suitable for vehicles in which the lower ISOFIX anchor point is **no higher than 3 cm** above the seat surface of the vehicle seat (Fig. 2) from MA footrest adapter).

Please check the dimensions in your vehicle before installation.

The footrest holder with Seatfix can only be used in conjunction with an adapter plate (Fig. 3). It provides additional holders for other accessories:

- **Footrest or support foot**

You can find out what options your rehab child car seat offers in the instructions for use for your rehab child car seat or under this link: www.thomashilfen.com/product-instructions

Fitting the adapter plate for footrest with SEATFIX in the vehicle

Make sure that an adapter plate is fitted to your rehab car seat (Fig. 3)

Place the footrest adapter with SEATFIX in your vehicle in front of the lower ISOFIX anchor points. (Fig. 4). Pull out the two SEATFIX connectors of the footrest adapter as far as possible by first pressing and holding the small button and then pressing the large button at the same time (Fig. 5+6).

Place the connectors on the lower anchor points and lock the connectors. You will hear a clear „click“ and the display changes from red to green. (Fig. 7 + 8) with detail)

Check the indicator windows (Fig. 8) on both sides to ensure that the connectors are correctly locked. The colour marking is now „green“ and the footrest adapter with SEATFIX can no longer be pulled forwards.

Caution: If you see a red mark in the display windows, the connector are not yet correctly locked. If the footrest mount with SEATFIX can still be pulled forwards, the connectors are not yet correctly locked.

Mounting your rehab child car seat on the adapter plate for footrest with SEATFIX

Tip: Before fitting in your vehicle, please push the front vehicle seat far forwards. This will give you more space in the vehicle for installation.

Now place the rehab child car seat in the centre of the footrest adapter with Seatfix. To do this, use the groove in the adapter plate (Fig. 9) and the stop in the footrest holder (Fig. 10). For a secure connection, press your rehab car seat firmly onto the footrest adapter with Seatfix.

Now push your installed rehab child car seat as close as possible to the backrest of your vehicle. The rehab child car seat must have as much contact as possible with the backrest of your vehicle.

Removing your rehab child car seat from your vehicle

If your rehab car seat is secured with a top tether, release this connection first. Pull the top end of the rehab child car seat forwards. (Fig. 11) This will release the Velcro connection and you can remove the rehab car seat from your vehicle (Fig. 12).

Removing the Adapter plate for footrest with SEATFIX in the vehicle

Next, release a Seatfix connector of the footrest adapter with Seatfix by first pressing and holding the small button and then pressing the large button (Fig. 5). Now push the detached connector back into the footrest holder. Then release the second side and push this connector back into the footrest holder.

The footrest adapter with Seatfix can now be removed from your vehicle.

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG · Walkmühlenstr. 1 · D - 27432 Bremervörde · Germany