AVOVA Sperling Reha • Axxkid NEXTKID Reha Recaro Monza CFX Reha

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Footrest (high-tech base)

Assembly instructions – Accessory

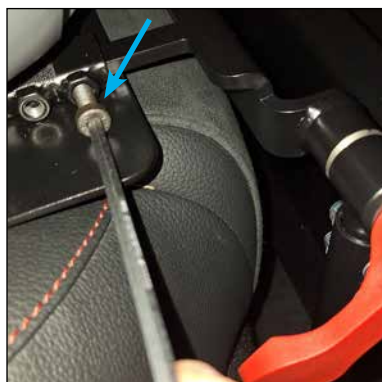
1

2

3

4

5

6

7

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
8a

8b

9

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This footrest (Fig. 1) can only be used in conjunction with one of the following accessories combination:

- **adapter plate + adapter plate for footrest**
- **adapter plate + adapter plate for footrest with seatfix**
- **adapter plate + tilting base**
- **adapter plate + swivel base**
- **adapter plate + swivel and tilting base**

You can find out what options your rehab child car seat offers in the instructions for use for your rehab child car seat or under this link: www.thomashilfen.com/product-instructions

You will need the two allen keys size 3 and size 4 included with the adapter plate for assembly/disassembly.

Mounting the footrest to your rehab child car seat

Firstly, install your rehab child car seat with the corresponding accessories in your vehicle. Follow both the instructions for use for your rehab child car seat and the installation instructions for your accessories.

Tip: First loosen the excenter lever (Fig. 2) and the footrest in your vehicle to slide the guide rails into the accessory holders. This keeps the footrest easy to handle.

Slide the guide rails into the holder of the accessories of your rehab child car seat. (Fig. 3)

The footrest should be positioned as close as possible to the edge of your vehicle seat and as vertically as possible. Tighten one screw on each side to fix the footrest in position. Use the size 4 allen key (Fig. 4) for this.

Contact surface for the footrest in the vehicle floor

Next, turn the legs of the footrest out far enough to create a secure contact surface in the vehicle floor. (Fig. 5) To fix the length, screw the nut (Fig. 6) against the lower end of the leg.

Height of the footrest

Fold down the footrest. Adjust the height of the footrest to the height of the user. Loosen the screws using the allen key size 3 (Fig. 7). Position the footrest at the required height and retighten the screws.

The footrest can be folded up for the user to get on. (Fig. 8)

Fixing the footrest

If you do not want to use the folding function of the footrest, you can fix the footrest in the 4-sided hole of the footrest using the screw and nut and a 10 mm open-end spanner (Fig. 9).

Disassembly the footrest

Loosen the two screws on the footrest holder (Fig. 3) to be able to remove the footrest again.

Tip: The footrest becomes easy to handel if you fold the guide rails down again by loosening the excenter levers (Fig. 2). Tighten the excenter levers again and fold up the footrest so that the magnet holds the footrest in position.

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG · Walkmühlenstr. 1 · D - 27432 Bremervörde · Germany