AVOVA Sperling Reha • Axxkid NEXTKID Reha Recaro Monza CFX Reha

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Stand (High-tech base)

Assembly instructions – Accessory

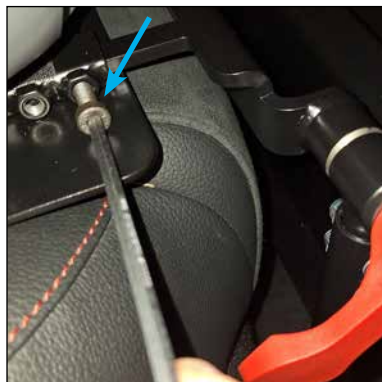
1

2

3

4

5

6

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This support leg (Fig. 1) can only be used in conjunction with one of the following accessories combination:

- **adapter plate + adapter plate for footrest**
- **adapter plate + adapter plate for footrest with seatfix**
- **adapter plate + tilting base**
- **adapter plate + swivel base**
- **adapter plate + swivel and tilting base**

You can find out what options your rehab child car seat offers in the instructions for use for your rehab child car seat or under this link: www.thomashilfen.com/product-instructions

You will need the allen key size 4 included with the adapter plate for assembly/disassembly.

Fitting the stand to your rehab child car seat

Firstly, install your rehab child car seat with the corresponding accessories in your vehicle. Follow both the instructions for use for your rehab child car seat and the installation instructions for your accessories.

Tip: First loosen the excenter lever (Fig. 2) in your vehicle to slide the guide rails into the accessory holders. This keeps the support leg easy to use.

Slide the guide rails into the holder of the accessories of your rehab child car seat. (Fig. 3)

The stand should be positioned as close as possible to the edge of your vehicle seat and as vertically as possible. Tighten one screw on each side to fix the support leg in position. (Fig. 4) Use the allen key size 4 for this.

Support for the stand in the foot space

Next, turn the legs of the stand out far enough to create a secure support in the foot space. (Fig. 5) To fix the length, screw the nut (Fig. 6) against the lower end of the leg.

Dismantling the footrest

Loosen the two screws on the footrest holder (Fig. 3) to be able to remove the support leg again.

Tip: The support leg becomes easy to use if you fold the guide rails down again by loosening the excenter levers (Fig. 2). Tighten the eccentric levers again.

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG · Walkmühlenstr. 1 · D - 27432 Bremervörde · Germany