Axkid ONE 2 Reha

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2. Preface / Manufacturer

Dear Customer,

Thank you for choosing our product and for the trust you have placed in us. In order for you and the user to enjoy your new product safely, conveniently, and comfortably, we kindly ask that you read the Instructions for Use. Doing this will also help avoid damage, injury, and incorrect operation and handling of the product. Please ensure that a copy of the Instructions for Use remains with the product so that every user has access to it.

The modification to the rehab car seat changes the handling of the car seat in a few points; there may be deviations from the consumer product.

If you have any questions, please contact your authorised distributor or us directly.

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Phone: +49 (0) 4761 8860



www.thomashilfen.de

We reserve the right to make technical changes. This can lead to differences between your product and how it is presented in brochures and other media.

Table of Contents

1. Figures	3
2. Preface / Manufacturer	26
3. Labels / Model Identification	27
4. General Remarks / Safety Instructions	28
5. Intended Purpose	29
5.1 Intended Use / Application Areas	29
5.2 Indication / Patient Target Group / Intended Users.	29
5.3 Risks Associated with Use and Contraindications	29
6. Product Description	29
6.1 Scope of Delivery	29
6.2 Standard Equipment	29
6.3 Putting into Service.	29
6.4 Teaching the User	29
7. Adjustment Options / Features / Accessories	30
8. Components	31
8.1 Permitted and unauthorised seating positions of the Axxid ONE 2 Reha child seat	33
8.2 Installing the Axxid ONE 2 Reha in the vehicle	33
8.3 Recline adjustment of the Axxid ONE 2 Reha child seat	34
8.4 Seat adjustments.	34
8.5 Adjusting / Removing the Axxid ONE 2 Reha from your vehicle	35
9. Technical Data	36
10. Material Specifications	36
11. Cleaning, Maintenance, and Repair.	37
11.1 Cleaning, Care Instructions, and Disinfection on a Daily Basis	37
11.2 Maintenance and inspection by the distributor / Maintenance Schedule.	38
11.3 Spare Parts	38
11.4 Storage / Disposal	38
12. EU Declaration of Conformity	38
13. Warranty	39
14. Period of Use / Service Life.	39
15. Reuse.	39
16. Distributor	39

3. Labels / Model Identification

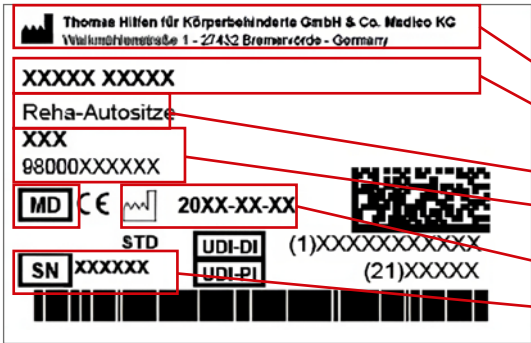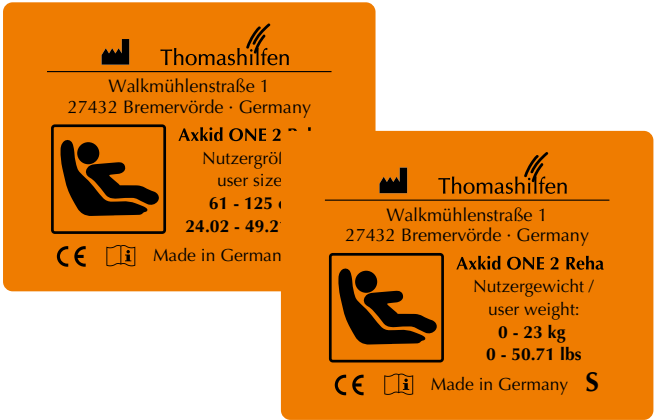Label	Meaning
	Barcode Label Position on the Product (Fig. 1) - Manufacturer - Product Name - Product Group - Item Number short / long - MD = Medical Device - Manufacture Date - Serial Number
	Type plate Position on the Product (Fig. 2) - Manufacturer - Product Name - User weight - CE marking - Follow Instructions for Use

4. Remarks

4.1 General Remarks

- Read the Instructions for Use carefully before first use. If you have difficulty reading, please have them read to you. If the copy of the Instructions for Use is missing, you can download one from www.thomashilfen.de. The contents of the digital Instructions for Use can be enlarged.
- Check the packaging for any transport damage and to be sure the product is in flawless condition.
- Changes to the product are only permitted within the scope of the intended adjustment options.
- Cleaning and daily care can be provided by relatives and caregivers. Maintenance and inspection are done by the distributor.
- In some countries it necessary to carry a medical certificate.
- The rehab car seat can be used on one of the rear seats and the front passenger seat. The selected car seat **MUST** be forward-facing and must have an Isofix connection.
- All labels have to remain on the product and should not be tampered with.

4.2 Safety Instructions

- Check at least every 3 months that the settings on the product still work for the user's current body measurements. If need be, have a distributor / orthopaedic technician adjust the settings.
- Do regular (weekly) checks to make sure that the setting options on the product work.
- Please do not overload the product and note the maximum user weight and the maximum load capacity (see technical data).
- Never leave the user unattended with the product.
- The product should only be used by one user at a time. No other individuals should be placed in or on the product.
- All users have to be familiar with how the product works. This way in an emergency the user can be removed quickly from the product.
- Do not leave the user unattended when getting in or out of the product.
- The user should always be positioned with the 5-point safety & positioning belt (basic equipment).
- Intense sunlight and heat can lead to high surface temperatures on the product. As a result, there is a risk of injury when touching the product. Only use the product once it has cooled down enough. In general, try to protect the product from direct sunlight.
- Please note the safety standards which apply to your vehicle as outlined in the vehicle's Instructions for Use to ensure the safety of the user.
- The Instructions for Use have to remain with the product.
- If there is a serious and unexpected accident with the product, please inform our company immediately and the responsible authorities.

5. Intended Purpose

5.1 Intended Use / Application Areas

The Axxkid ONE 2 Reha car seat is a class 1 medical device.

It is designed for transporting children with disabilities and for maintaining a secure seating position in motor vehicles.

The child is always secured with the 5-point safety belt of the rehab car seat.

The 5-point belt of the rehab car seat also serves to position and support the child in an upright sitting position.

Sitting position of the child. The rehab car seat also has an integrated seat tilt.

The rehab car seat provides additional support for safe sitting, stabilisation of head control and a stable seat in the pelvic area. The product also offers accessories for customisation. The rehab car seat is used as a rear-facing reboarder.

5.2 Indication / Patient Target Group / Intended Users

The rehab car seat is for children with structural and / or functional limitations or poor head and torso posture. It provides a stable, physiological position if a standard child car seat does not securely support the child while seated in the vehicle.

5.3 Risks Associated with Use and Contraindications

If used properly, risks can be eliminated during use. There are no known contraindications.

6. Product Description

6.1 Scope of Delivery

Your product is delivered with the standard equipment and comes preassembled. Any accessories ordered along with the product are included and have to be assembled subsequently. A copy of the Instructions for Use is enclosed with the product.

6.2 Standard Equipment

Seat shell (directed backwards); seat and backrest with 12-position height-adjustable headrest; head cushion; seat cushion; integrated 5-point-belt with central adjuster; height-adjustable shoulder belt with Axxkid ONE 2 Reha belt padding system incl. crotch pad (small); side impact protection (ASIP); carrying handle; adjustable resting and sleeping position; removable trunk and hip guide pads; removable and washable covers; stand base; automatic rebound bar with built-in sound; abduction block; ISOFIX-Connectors

6.3 Putting into Service

- Carefully remove the transport protection and the packaging.
- Keep the packaging material and small parts out of the reach of children to prevent swallowing or choking.

6.4 Teaching the User

The user is taught how to use this device by trained staff who will reference these Instructions for Use.

Trained staff should install the seat in the vehicle with the user and adjust the seat to the measurements of the actual user.

The user has to be shown all of product's functions as well as how to install and remove it from the vehicle.

7. Adjustment Options / Features / Accessories

Please note:

The Axxid ONE 2 Reha is an extended i-Size child car seat. It is approved according to the UN R129 directive for use in i-Size compatible seats in the car according to the manual of the respective car manufacturer. If in doubt please contact the child car seat manufacturer (www.axkid.com) or your dealer.

The Axxid ONE 2 Reha is approved as a restraint system for special requirements and is therefore marked with an "S".

7.1 5-point belt

For more information on the use of belts please consult the RECARO instructions for use provided with the seat.

Adjusting the belt length:

Slightly lift the seat cover and release the fabric flap by opening the hook and loop. Now you can see the central belt adjuster in the seat shell. To change the belt length, press the lever down and adjust the belt by either pulling the strap coming out of the front of the seat (shortening) or by pulling out both shoulder belts (lengthening). (Page 6, Fig. 7-8)

Belt buckle:

Press the red dent of the buckle to open. Separate the two upper belt fasteners. To close the belt, bring the two upper belt fasteners together, insert them into the buckle from above and push together until they lock in place with an audible „Click“. (Page 6, Fig. 8-9)

7.1.1 Lateral trunk support

The thorax pads can be individually positioned in height and depth on the lateral velour surfaces. (Fig. 1)

7.1.2 Lateral seat support

To attach the thorax pelotte pad, position it loosely in the seat and then bring the two Velcro fasteners on the side seat cushion together and fasten them there. (Fig. 2)

7.2 Accessories

Remarks: Installation information is found in the Instructions for Use, which are enclosed with the accessory. If the copy of the Instructions for Use is missing, you can download one from www.thomashilfen.de. The assembly / disassembly of the accessories is done with standard tools.

7.2.1 Head cushion

The additional head cushion serves to reduce the size of the headrest and gives smaller users more support. (Fig. 3)

7.2.2 Anti-escape vest

The anti-escape vest prevents the child from squirming out of the car seat. Dangerous spaces between the belt and the child will be discovered which the child might take advantage of to slip off the belt. (Fig. 4)

7.2.3 Car seat protection mat with back

The car seat protection mat is under the child's car seat on the seat of the vehicle and the seatback. It protects the seat surface and the seatback of the vehicle from damage. Small items can be stored in the front pockets. (Fig. 5)

7.2.4 NapUp


It is an easy-to-fit head support that can be retrofitted to almost any Thomashilfen rehab car seat. The NapUp provides a secure hold with minimal forehead contact and offers maximum freedom of movement.

To fit the NapUp, the belt straps are fed through the belt guides of the rehab seat and fastened if possible. If the shoulder belt guides do not offer sufficient space, the straps are routed around the headrest and fastened. (Fig. 6)

8. Components

Shoulder straps (A)	Headrest adjustment (M)
Seat belt buckle	Legroom adjustment knob (N)
Roll bar release (C)	Rear handle (O)
Roll bar (D)	Support foot position indicator (P)
Head restraint (E)	Support foot release strap (Q)
Eye level marker (F)	Support foot (R)
ASIP pad (G)	Sound indicator (S)
Tilt adjustment knob (H)	Belt release button (T)
Support foot adjustment knob (I)	Belt tensioner (U)
AASIP connector (J)	ISOFIX locking indicator (V)
Roll bar button (K)	ISOFIX button (X)
Side cover (L)	ISOFIX connectors (Y)

Parts

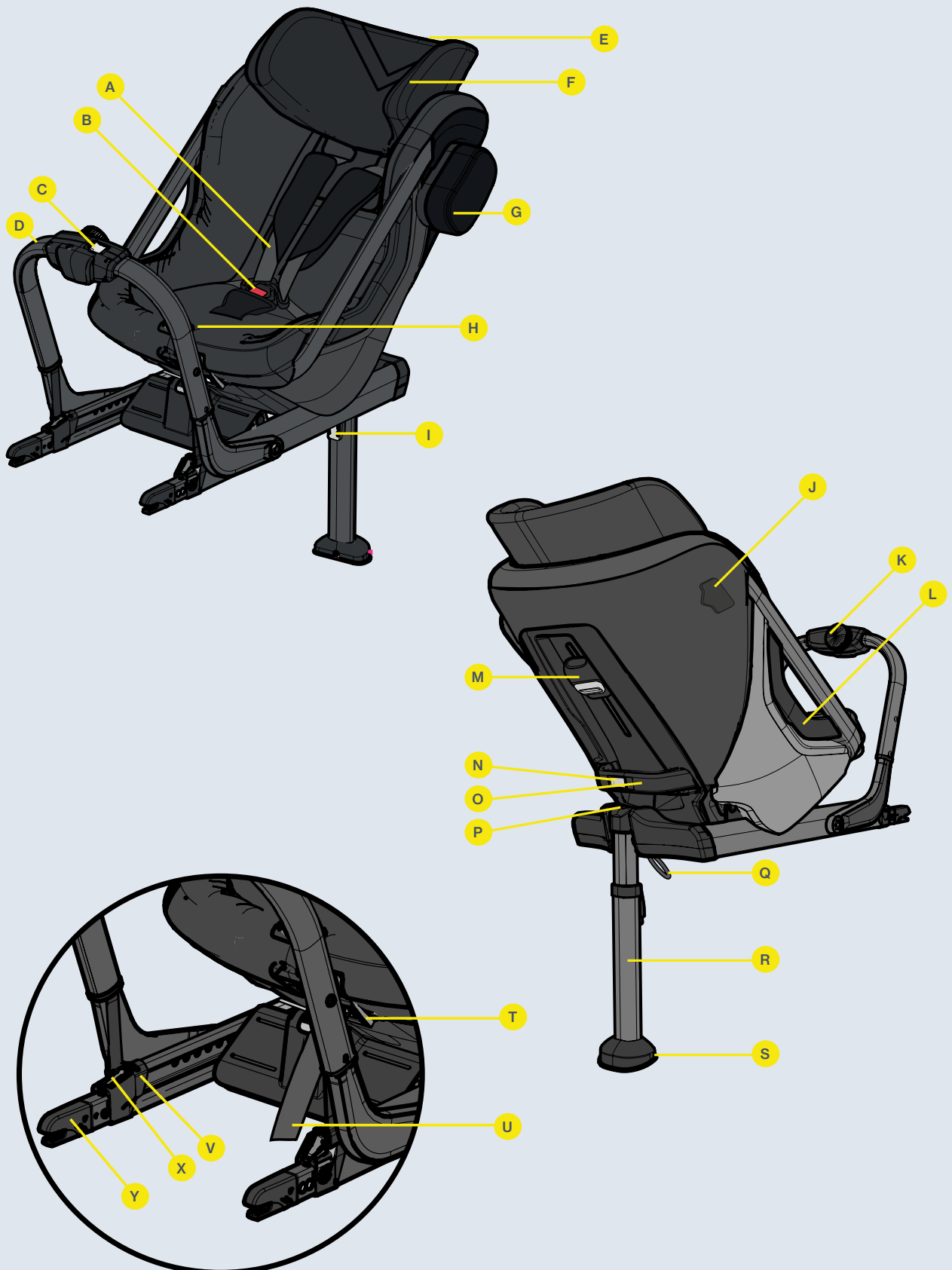8.1 Allowed and disallowed positioning of Axkid ONE 2 Reha car seat

Axkid ONE 2 Reha car seat can only be installed rear-facing using ISOFIX. It is approved for use in all i-Size approved seating positions but may also fit in other non i-Size-approved ISOFIX seating positions. Check your vehicle instruction manual to find suitable seating positions and check the vehicle list on www.axkid.com/ www.thomashilfen.de.

Front passenger seat with ISOFIX (no active airbag)	YES/NO*
Front passenger seat with ISOFIX (active airbag)	NO
Rear passenger seat with ISOFIX	YES
Side and rear facing seats with ISOFIX	NO
Seats without ISOFIX	NO

IMPORTANT:

If you wish to install your Axkid ONE 2 Reha car seat in a location where a frontal airbag is fitted, the airbag must be disconnected according to your vehicle manufacturer instruction. Note that some vehicles require an authorized service center to disconnect the airbag.

* The General Regulation of Traffic in Spain, in its article 117, establishes the prohibition of driving with children of stature equal to or less than 135 centimeters located in the front seats of the vehicle, without exceptions.

Never install a child restraint in a passenger seat where an active airbag is fitted.

8.2 Installing Axkid ONE 2 Reha car seat in the vehicle

- Before installing Axkid ONE 2 Reha car seat in your vehicle, minimize its size by releasing the legroom adjustment button (N) and slide the seat towards the front brace (D). Fold down the support-leg (R) until a click can be heard and the support-leg position indicator (P) turns green.
- Release the front brace release button (C) and fold the front brace (D) towards the seat. Keep holding the front brace (D) with one hand and grab the back handle (O) with the other hand and lift the Axkid ONE 2 Reha car seat into your vehicle with the front brace (D) first and then turn it to a rear-facing position.
- Push the ISOFIX connectors (Y) into the ISOFIX connection points in the vehicle until you hear a click and both ISOFIX locking indicators (V) turn green.
- Let go of the front brace (D) which will start tensioning itself against the vehicle seat. When it stops, turn the front brace knob (K) clockwise until you feel and hear a click, the front brace (D) should now be firmly pressed against the vehicle seat. The front brace knob (K) has a built-in torque limiter and the click indicates that the front brace (D) is tightened correctly and cannot be tightened any more.

- 5 Push the support-leg button (I) and pull the support-leg (R) downwards until it is in contact with the vehicle floor. Let go of the support-leg button (I) and lift the car seat upwards to further extend the support-leg (R) until you hear a click. The support-leg (R) should now be locked and firmly pressed against the vehicle floor. Note: The Axxid ONE 2 Reha car seat is equipped with a beeping sound indicator (S) that will keep beeping until the support-leg (R) is firmly pressed against the floor, indicating a correct installation. green.
- 6 Push the ASIP pad (G) into the slots on the ASIP connector (J) and slide downwards, the ASIP pad (G) will click into place. If there is a plastic cover on the ASIP connector (J), remove it and place it on the ASIP connector (J) on the opposite side of the seat before fitting the ASIP pad (G). Make sure the ASIP pad (G) is attached on the side of Axxid One car seat that is closest to the vehicle door.

8.3 Restraining your child in Axxid ONE 2 Reha car seat

Before restraining your child in the Axxid ONE 2 Reha car seat, make sure you have the correct insert installed in the seat. Axxid ONE 2 Reha is designed to grow with your child and to ensure a comfortable and safe fit for all children. Axxid ONE 2 Reha recommends using the relevant insert until it is too tight for your child, but to a minimum length according to table below (note that the age is just an indication):

- 7 Loosen the harness by pressing the harness release button (T) and pull out both shoulder straps (A) at the same time. Avoid pulling only one strap at the time since it may cause the shoulder straps (A) to become uneven
- 8 Unlock the buckle by pressing the buckle release button (B) and place the straps outside of the seat.
- 9 Place your child in the seat and place the shoulder straps (A) over your child's shoulder. Lock the harness by first positioning the parts of the buckle that is attached to the shoulder straps (A) together and then push them into the buckle until you hear a click.
- 10 Adjust the headrest (E) until the eye level indicators (F) is in line with your child's eyes by pushing both headrest adjustment buttons (M) and slide it upwards or downwards.
- 11 Check that the shoulder straps (A) is going through the seat straight from your child's shoulders or maximum 1 cm above. You might need to adjust the headrest (E) again, even if it makes the eye level indicator get out of line with your child's eyes. Now pull on the shoulder straps (A) to tighten the harness over your child's hips. While keeping tension in the shoulder straps (A), pull the harness tension strap (U) until the harness is tightened enough, you should not be able to fit more than one finger between the shoulder straps (A) and your child's shoulders.

8.4 Seat adjustments

After the Axkid ONE 2 Reha car seat has been installed in your vehicle you can easily adjust both the legroom and the recline position for your child. This can also be done while your child is restrained in the car seat. To ensure a good fit in all cars and give the best protection for your child, Thomashilfen strongly recommends that you use the guided positions marked on the side of the car seat. As your child grows it is important to adjust the recline to a more upright position to give the best possible protection for your child's neck in the case of an accident. The size indication is based on your child's length. Make sure both arrows points at the correct zone.

Age	Adjustment	
Toddler 0-3 years	All recline positions Several legroom positions	61-105 cm
Toddler 3-7 Jahre	All recline positions Several legroom positions	106-125 cm

8.5 Adjusting / Removing Axkid ONE 2 Reha car seat from your vehicle

- The legroom is adjusted by releasing the legroom adjustment button (N) and sliding the seat forward or backwards. To avoid Axkid ONE 2 Reha from accidentally sliding forward, grab the back handle (O) and gently pull it backwards before releasing the legroom adjustment button (N).
- The recline position of the car seat can be adjusted by turning the recline position knob (H).
- Push the support-leg button (I) and collapse the support-leg (R) to its shortest position.
- Slide the front brace release button (C) and fold the front brace (D) towards the seat to release the tension against the vehicle seat.
- Disconnect the ISOFIX connectors (Y) by pushing down on the ISOFIX buttons (X) and then slide them backwards, away from the vehicle seat.
- Keep holding the front brace (D) with one hand and grab the back handle (O) with the other hand and lift Axkid ONE 2 Reha car seat out of your vehicle.
- Fold up the support-leg (R) by pulling the support-leg release strap (Q) and folding the support-leg (R) upwards.

9. Technical Data

Axxid ONE 2 Reha

Seat depth (to crotch strap)	16 cm
Seat depth (to front edge)	28 cm
Seat width	21 - 29 cm
Back height (seat inside)	52 - 63 cm
Back height	29 - 63 cm
Shoulder height (with belt guide)	26 - 37 cm
Seat tilt (basic model)	45° / 65°
Overall dimensions (W x H x D)	44 x 76 x 54 cm
Weight of the seat (basic model)	9.9 kg
User height	61 - 125 cm
Max. user weight	23 kg

10. Material Specifications

The upholstery material used is "flame-retardant", tested in accordance with DIN EN 1021-1 and DIN EN 1021-2. The degree of resistance to ignition of materials and assemblies was tested in accordance with the standard and assessed as harmless. The 100% polyester fabric is a high-quality, breathable fabric and has been positively tested for biocompatibility. (DIN EN ISO 10993-5 and 10993-10)

11. Cleaning, Maintenance, and Repair

11.1 Cleaning, Care Instructions, and Disinfection on a Daily Basis


11.1.1 Padding

The seat textile cover can be machine washed at 40° C on a gentle cycle. Do not put the cover in a dryer as this can damage the cover and the padding may separate from the fabric. Visit www.thomashilfen.de to find videos explaining how to remove and attach the cover.

To remove the textile cover from the seat, pull outwards on the side cover (L) and de-attach the Velcro. Open the zippers at the backrest and gently remove the cover from the seat. Make note how it is attached around the seat to make it easier to put back. When reinstalling the side cover (L), insert it under the aluminum frame and simply push back into the snap attachment. To remove the headrest cover, open the zip fasteners at the back of the headrest and carefully remove the cover.

If the seat cover needs replacing, ensure only original products from Thomashilfen are used. If other products are used, Axxid ONE 2 Reha car seat's safety system may be compromised and could lead to severe injury in case of an accident.

The Axxid ONE 2 Reha

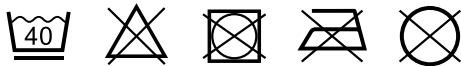We would like to point out that even high-quality materials can fade as a result of long and intense exposure to sunlight or frequent washing.

11.1.2 Hook and Loop Fasteners

To keep the hook and loop fasteners working well, clean them now and then with a brush. Always close or cover the hook and loop fasteners before washing.

11.1.3 Plastic Parts

The plastic parts on the product can be cleaned and disinfected with standard soap (neutral soap).

11.1.4 Metal Parts

The metal parts on the product can be cleaned and disinfected with standard soap (neutral soap).

11.1.5 Disinfection

Clean the padding, plastic parts, and metal parts before disinfecting.

Damp wipe all the plastic and metal parts on the product with disinfectant. Only water-based products should be used. When disinfecting note the directions from the manufacturer in question.

11.2 Maintenance and inspection by the distributor

11.2.1 Preparation

A copy of the Instructions for Use has to be enclosed with the product. If one is not available, request a copy from the distributor. Familiarise yourself with how the product works. If you are not familiar with the product, read the Instructions for Use before testing it. Clean the product before starting to test it. Please note the care instructions in the Instructions for Use and product-specific test instructions. If you have further questions, please contact the staff member you dealt with or Thomashilfen directly.

Service and repairs to the product should only be done by qualified staff. It is the user's responsibility to ensure that the prescribed inspections / maintenance are done regularly and in a timely fashion. If damage is apparent, the user has to actively inform and have the damage fixed immediately by qualified staff.

11.2.2 Maintenance Schedule – Annual Inspection

	Area	Function		Damage		Replace	
1	Plastic body	👁		👁			
2	Adjustment options	👁		👁			
3	Padding			👁			
4	Belts	👁		👁			
5	Plastic parts	👁		👁			
6	Screws and nuts	👁		👁			
7	Labels			👁			
	Accessory						
8	Padding			👁			
9	Belts	👁		👁			
10	Adjustment options	👁		👁			
11	Plastic parts	👁		👁			
12	Metal parts	👁		👁			
13	Screws and nuts	👁		👁			
done by:				on:			
👁 = Inspection 🔄 = Reuse ✓ = Test completed							

11.3 Spare Parts

Only original spare parts should be used for repairs. A list of the spare parts is available at www.thomashilfen.de. To receive the right spare parts or for possible technical recalls, we normally require the serial number / UDI-PI and the exact UDI-DI.

Removal / installation of spare parts should only be done by qualified staff! In the event of returns, please make sure the product is securely packaged for transportation.

11.4 Storage

Store the product in a safe place when it is not being used. Do not lay or put any heavy objects on the product while it is being stored. Protect the product from weather like rain, intense sunlight, and frost. If the product is wet, wipe the metal and plastic parts dry and let the padding dry before storing the product.

11.5 Disposal

If the product is no longer usable due to its condition or it has reached the end of its service life, it can be recycled at the local waste disposal company. Please refer to the local guidelines for recycling and disposal.

12. EU Declaration of Conformity

We hereby declare as the party solely responsible that this product and any versions and accessories comply with the basic regulations. The current EU Declaration of Conformity can be downloaded at www.thomashilfen.de or requested from us directly.

13. Warranty

Thomashilfen offers you the two-year warranty as required by law for this product. The warranty period starts when the item is handed over. The warranty includes all claims which impact function. Not included is damage which is the result of improper use (e.g. overloading) and natural wear and tear.

14. Period of Use / Service Life

The product has a period of use of up to 6 years if used correctly and serviced regularly by a professional as per the information from the manufacturer. The product can be used beyond this point if it is in safe and technically sound condition.

The theoretical period of use does not equate to a guaranteed service life. It depends on the intensity of use and always undergoes an individual assessment by the distributor.

The period of use ends before the given time frame if the user has reached the maximum body weight or maximum body height to safely use the product.

The period of use does not apply to wear parts like padding, covers, etc.

The product has a service life of 8 years if the requirements stipulated for the period of use are met. The service life consists of the period of use and storage times.

15. Reuse

The product is not suitable for reuse. The functionality of child car seats cannot be determined given a theoretical change of users. There could be damage to the product that is not visible which rules out continued safe use. The sale and purchase of used child car seats is therefore ruled out by the manufacturer.

16. Distributor

Astris PME Pty Ltd

Young & Canberra Office:
Astris PME Pty Ltd
5 McVeigh St
Young NSW 2594

Sydney Office:
Astris PME Pty Ltd
30 Prime Dr
Seven Hills NSW 2147

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Tendercare Ltd.
Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG