


Hercules Small



Armlehnen + Unterarmauflage

Montageanleitung – Zubehör

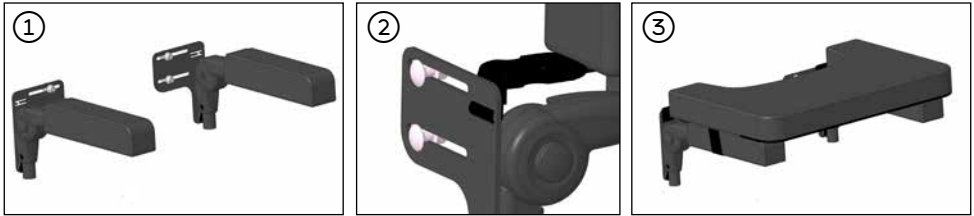

Hinweis: Sollten Armlehnen und Unterarmauflage verwendet werden, führen Sie zuerst Schritt 1 aus dem Abschnitt Unterarmauflage aus! Die Armlehnen müssen vom Sitz getrennt sein.

Armlehnen

Art.-Nr.: 9800004890000

Entfernen Sie die hinteren Abdeckpolster vom Hercules Rücken, danach setzen Sie die Grundplatten der Armlehnen auf die Rückenplatte auf. Führen Sie von hinten die Schrauben durch die Langlöcher und kontern diese mit den Muttern von der Vorderseite. Die Armlehnen sind in Höhe sowie Breite variabel einstellbar. Nach der Einstellung, bringen Sie die Abdeckpolster wieder an. (Bild 1)

Unterarmauflage

Art.-Nr.: 9800004910000

Zunächst befestigen Sie die kurzen Gurtenden an den Grundplatten der Armlehnen. Befestigen Sie dazu die Gurtschlaufen, der kurzen Gurtenden in den H-Nuten der Trägerplatten. (siehe Bild 2)

Anschließend montieren Sie die Armlehnen, wie zuvor beschrieben. Danach führen Sie die Gummibänder der Tischplatte über die Armlehnen und stecken die hinteren Verschlussschnallen in die an den Armlehnen befestigten Gegenstücke ein (siehe Bild 3).

Zum Schluss können Sie die Unterarmauflage mit den vorderen Gurtbändern festziehen.


Thomashilfen




Mitglied in der Internationalen
Födergemeinschaft für Kinder-
und Jugendrehabilitation e.V.

Hercules Small



Armrests + Table

Assembly instructions – Accessory

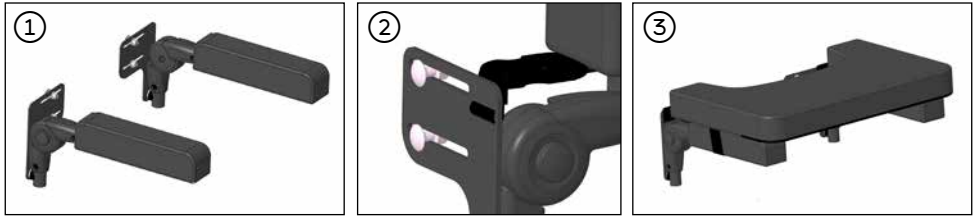

Attention: If armrests and table are used, first carry out step 1 from the table section!
The armrests must be separated from the seat.

Armrests

Art.-Nr.: 9800004890000

Remove the rear cover pads from the Hercules back, then place the base plates of the armrests on the back plate. Insert the screws from the rear through the slotted holes and lock them with the nuts from the front. The armrests are adjustable in height and width. After adjustment, replace the cover pads. (fig. 1)

Table

Art.-Nr.: 9800004910000

First attach the short strap ends to the base plates of the armrests. To do this, attach the belt loops of the short belt ends in the H-slots of the support plates. (Fig. 2)

Fit the armrests as described above. Guide the elastic straps of the table top over the armrests and insert the rear buckles into the counterparts attached to the armrests. (Fig. 3)

The forearm support can now be tightened with the front straps.


Thomashilfen



Mitglied in der Internationalen
Födergemeinschaft für Kinder-
und Jugendrehabilitation e.V.