# Axkid NEXTKID Reha Recaro Monza CFX Reha

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The abduction guide (Fig. 1) **can only** be used in **conjunction with an adapter plate** (Fig. 2) for the following rehab car seats:

- **Nextkid - rehab child car seat**
- **CFX Reha-rehab child car seat**

You can find out what options your rehab child car seat offers in the instructions for use for your rehab child car seat or under this link: [www.thomashilfen.com/product-instructions](http://www.thomashilfen.com/product-instructions)

You will need the size 3 allen key included with the adapter plate for assembly/disassembly.

## Fitting the abduction guide to your rehab child car seat

Make sure that an adapter plate is fitted to your rehab car seat (Fig. 2)

Loosen the star grip screw (Fig. 3) and pull the square profile out of the angle of the abduction guide. To install the seat, place the child car seat on its back (Fig. 2)

Slide the bracket between your rehab child car seat and the adapter plate (Fig. 4). Screw the bracket into the middle slot from below using the two washers and two screws. (Fig. 5) You can adjust the position in the slotted hole to suit your child's needs.

Now loosen the grub screw underneath the abduction guide (Fig. 6) and pull the square profile out to a maximum of 8 cm (Fig. 7). Tighten the grub screw again. Insert the square profile into the bracket holder until the abduction guide rests on the seat surface of your rehab child car seat. Secure the abduction guide with the star grip screw. (Fig. 3)

## Removal of the abduction guide for easier getting in and out

To make it easier to get your child into or out of the rehab child car seat, you can remove the abduction guide quickly and easily.

Loosen the star grip screw at the bottom of the bracket (Fig. 3) to remove the abduction guide. Once you have positioned your child in the rehab car seat, you can reinsert the abduction guide into the square holder and secure it with the star grip screw.

**Tip:** If you also use the seat depth extension in addition to the abduction guide, you must push the cover of the seat depth extension to the side to gain free access to the square support. (Fig. 8)

## Cleaning

First remove the abduction guide from the bracket (Fig. 3). Now you can open the elastic band of the cover (Fig. 9) and easily remove the cover of the abduction guide. Observe the following washing symbols for cleaning the cover:

## Removing the abduction guide from your rehab child car seat

Loosen the two screws with the allen key size 3 on the adapter plate to remove the bracket with the abduction guide.

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG · Walkmühlenstr. 1 · D - 27432 Bremervörde · Germany