AVOVA Sperling Reha • Axxid NEXTKID Reha Recaro Monza CFX Reha

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The seat depth extension **can only be used in conjunction with an adapter plate** (Fig. 2) for the following rehab car seats:

- **Axkid Nextkid Reha** - child car seat
- **Recaro Monza CFX Reha** - child car seat

You can find out what options your rehab child car seat offers in the instructions for use for your rehab child car seat or under this link: www.thomashilfen.com/product-instructions

You will need the size 3 Allen key included with the adapter plate for assembly/disassembly.

Fitting the seat depth extension to your rehab child car seat

Slide the bracket between your rehab child car seat and the adapter plate (Fig. 3). Screw the bracket from below with the 4 washers and 4 screws. (Fig. 4)

The elongated holes allow you to adjust the seat depth to your child's needs.

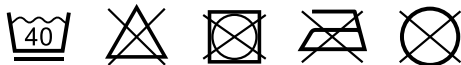
Tip: If you also use the abduction guide in addition to the seat depth extension, you must push the cover of the seat depth extension to the side in order to gain free access to the square support for the abduction guide. (Image 5)

Removing the seat depth extension

Loosen the four screws with the Allen key size 3 on the adapter plate to remove the seat depth extension.

Cleaning

The cover of the seat depth extension is easy to remove once you have opened the Velcro fastener. Please note the following washing symbols for cleaning the cover:

To maintain the functionality of the Velcro fasteners, brush the Velcro straps with a brush from time to time. Always close or cover the Velcro fasteners when washing.

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG · Walkmühlenstr. 1 · D - 27432 Bremervörde · Germany