AVOVA Sperling Reha • Axxid NEXTKID Reha Recaro Monza CFX Reha

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This turn and tilt plate (Fig. 1) can only be used in conjunction with an adapter plate (Fig. 2).

It offers additional holders for further accessories:

- **Footrest or support foot**
- **Stabilizing bow**

You can find out what options your rehab child car seat offers in the user manual for use for your rehab child car seat or under this link: www.thomashilfen.com/product-instructions

Mounting the swivel and tilting base in your vehicle

Make sure that an adapter plate is fitted to your rehab car seat (Fig. 3). Check the seat surface in the vehicle on which you wish to fit your rehab child car seat. If the seat modules are lower than 2 cm (Fig. 4), we recommend the cushion for the swivel plates to protect your vehicle seat. The cushion for the swivel plate is also useful if you have soft vehicle seats in your vehicle.

Place the swivel and tilt plate (Fig. 2) in your vehicle on the vehicle seat in front of the lower anchor points (Fig. 3). Pull out the two SEATFIX connectors of the swivel and tilt plate as far as possible by first pressing and holding the small button and then pressing the large button simultaneously (Fig. 4+5).

Place the connectors on the lower anchor points and lock the connectors. You will hear a clear „click“. (Fig. 8)

Check the indicator windows on both sides to ensure that the connectors are correctly locked. The colour marking is now „green“ (Fig. 9) and the rehab car seat can no longer be pulled forwards.

Caution: If you see a red mark in the display windows, the connector is not yet correctly locked. If the turn and tilt plate can still be pulled forwards, the connectors are not yet correctly locked.

Installation of your rehab child car seat

Now place the rehab child car seat in the centre of the swivel and tilting base. To do this, use the groove in the adapter plate (Fig. 10) and the stop in the Swivel and tilting base (Fig. 11) To ensure a secure connection, press your rehab car seat firmly onto the swivel and tilt plate.

Now push the rehab child car seat as close as possible to the backrest of your vehicle. The rehab car seat must have as much contact as possible with the backrest of your vehicle.

Swivel function of your rehab child car seat

The swivel function makes it easier for you to put your child in and take them out of the rehab car seat. Press the release button (Fig. 12) to turn your rehab child car seat towards the door opening. (Image 13)

Simply turn your rehab child car seat back to the starting position. The rehab child car seat locks automatically.

Reclining function of your rehab child car seat

To change the tilt function, loosen the two knurled screws (Fig. 14) and fix them in the new position (Fig. 15).

To do this, it may be necessary to loosen the connectors and pull the tilt plate forwards slightly. Now lock the connectors again (check that they are locked correctly as described above). Once you have set the correct position, push the Swivel and tilting base back as close as possible to the backrest.

Removing your rehab child car seat

Firstly, set the tilt function to 0° (Fig. 14). Pull your rehab child car seat forwards at the top of the backrest (Fig. 12). This will release the connection of your rehab child car seat from the Swivel and tilting base and you can remove your rehab child car seat from your vehicle.

Removing the turntable and tilt plate from your vehicle

Next, release a SEATFIX connector by first pressing and holding the small button and then pressing the large button at the same time. (Fig. 6) Then push the connector back into the swivel and tilting base. Then release the second side.

The turn and tilt plate can now be removed from your vehicle.

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG · Walkmühlenstr. 1 · D - 27432 Bremervörde · Germany