AVOVA Sperling Reha • Axxkid NEXTKID Reha Recaro Monza CFX Reha

Drehplatte (High-Tech Unterbau)

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Swivel base (high-tech base)

Assembly instructions – Accessory

1

2

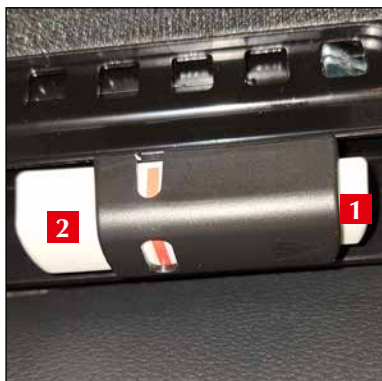
3

4

5

6

7

8

9

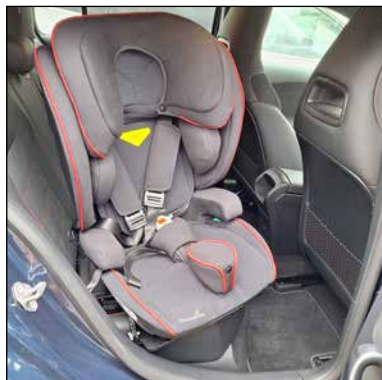
10

11




12

13

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14

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This swivel base (Fig. 1) can only be used in conjunction with an adapter plate (Fig. 2).

It offers additional holders for further accessories:

- **Footrest or Stand**
- **Stabilizing bow**

You can find out what options your rehab child car seat offers in the instructions for use for your rehab child car seat or under this link: www.thomashilfen.com/product-instructions

Mounting the swivel base in your vehicle

Make sure that an adapter plate is fitted to your rehab car seat (Fig. 2). Check the seat surface in the vehicle on which you wish to fit your rehab child car seat. If the seat modules are lower than 2 cm (Fig. 3), we recommend the cushion for the swivel plates to protect your vehicle seat. The cushion for the swivel plate is also useful if you have soft vehicle seats in your vehicle. (Fig. 4)

Place the Swivel base (Fig. 1) in your vehicle on the vehicle seat in front of the lower anchor points (Fig. 3). Pull out the two SEATFIX connectors of the swivel plate as far as possible by first pressing and holding the small button and then pressing the large button at the same time (Fig. 4 + 5).

Place the connectors on the lower anchor points and lock the connectors. You will hear a clear „click“. (Fig. 6)

Check the indicator windows on both sides to ensure that the connectors are correctly locked. The colour marking is now „green“ (Fig. 7) and the rehab child car seat can no longer be pulled forwards.

Caution: If you see a red mark in the display windows, the connector is not yet correctly locked. If the rotating plate can still be pulled forwards, the connectors are not yet correctly locked.

Installation of your rehab child car seat

Now place the rehab child car seat in the centre of the swivel base. To do this, use the cut out in the adapter plate (Fig. 10) and the limiter stop in the swivel base (Fig. 9) To ensure a secure connection, press your rehab child car seat firmly onto the swivel plate.

Now push the rehab child car seat as close as possible to the backrest of your vehicle. The rehab child car seat must have as much contact as possible with the backrest of your vehicle.

Swivel function of your rehab child car seat

The swivel function makes it easier for you to put your child in and take them out of the rehab child car seat. Press the release button (Fig. 10) to turn your rehab child car seat towards the door opening. (Fig. 13)

Simply turn your rehab child car seat back to the starting position. The rehab child car seat locks automatically.

Removing your rehab child car seat

Pull your rehab child car seat forwards at the top of the backrest (Fig. 14). This will release the connection of your rehab child car seat from the swivel plate and you can remove your rehab child car seat from your vehicle.

Removing the turntable from your vehicle

Next, release a SEATFIX connector by first pressing and holding the small button and then pressing the large button at the same time. (Fig. 6) Then push the connector back into the swivel plate. Then release the second side.

The turntable can now be removed from your vehicle.

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG · Walkmühlenstr. 1 · D - 27432 Bremervörde · Germany