AVOVA Sperling Reha • Axxid NEXTKID Reha Recaro Monza CFX Reha

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This inclination plate (Fig. 1) can only be used in conjunction with an adapter plate. (Fig. 2)

It offers additional holders for further accessories:

- **Footrest or support foot**
- **Stabilizing bow**

You can find out what options your rehab child car seat offers in the instructions for use for your rehab child car seat or under this link: www.thomashilfen.com/product-instructions

Mounting the tilting base in your vehicle

Make sure that an adapter plate is fitted to your rehab child car seat (Fig. 2)

Place the tilting base (Fig. 3) in your vehicle in front of the lower ISOFIX anchor points. (Image 3). Pull out the two SEATFIX connectors of the adapter plate as far as possible by first pressing and holding the small button and then pressing the large button. (Fig. 4 + 5).

Place the connectors on the lower anchor points and lock the connectors. You will hear a clear „click“ and the display changes from red to green. (Fig. 6+7)

Check the indicator windows (Fig. 7) on both sides to ensure that the connectors are correctly locked. The colour marking is now „green“ and the Adapter plate with Seatfix can no longer be pulled forwards.

Caution: If you see a red mark in the display windows, the connector is not correctly locked yet. If the footrest mount with SEATFIX can still be pulled tilting base forwards, the connectors are not correctly locked yet.

Installation of your rehab child car seat

Tip: Before fitting in your vehicle, please push the front vehicle seat far forwards. This will give you more space in the vehicle for installation.

Now place your rehab child car seat in the centre of the tilting base. To do this, use the cut out in the adapter plate (Fig. 8) and the limit stop in the reclining plate (Fig. 9) To ensure a secure connection, press your rehab child car seat firmly onto the tilting base.

Now push the tilting base as close as possible to the backrest of your vehicle.

The rehab child car seat must have as much contact as possible with the backrest of your vehicle.

Changing the tilting function

To change the tilting function, loosen the two star knob screws (Fig. 10). Lift the plate into the new position and secure the plate with the two star knob screws (Fig. 11).

To do this, it may be necessary to loosen the connectors and pull the tilting base forwards slightly. Now lock the connectors again, paying attention to the „click“ sound and the green markings. Now push the tilting base back as close as possible to the backrest.

Removing your rehab child car seat from your vehicle

Firstly, set the tilting function to 0° (Fig. 10)

If your rehab child car seat is secured with a top tether, release this connection.

Pull your rehab child car seat forwards at the top of the backrest. (Fig. 12) This will release the connection of your rehab child car seat from the tilting plate and you can remove your rehab child car seat from your vehicle.

Removing the tilting base from your vehicle

Next, release a SEATFIXconnector on the tilt plate by first pressing and holding the small button and then pressing the large button (Fig. 4). Now push the loosened connector back into the tilting plate. Then release the second side and push this connector back into the tilting plate as well.

Now the tilting base can be removed from your vehicle.

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG · Walkmühlenstr. 1 · D - 27432 Bremervörde · Germany